REF Set-1725 

REF Set-1771 

Praxis • Dental Surgery • Cabinet dentaire

Optimale Hohlkehhlpräparation - schnell - einfach - perfekt.
Optimal deep chamfer preparations - quickly - easily - perfectly.
 Préparation de moulure concave optimale - rapidité - simplicité - perfection.

Easy Chamfer zylindrisch, cylindrical, cylindrique

Easy Chamfer konisch, conical, conique

878SE-012C-FG 2x 

878SE-012F-FG 2x 

878SE-015C-FG 2x 

878SE-015F-FG 2x 

879SE-012C-FG 2x 

879SE-012F-FG 2x 

879SE-015C-FG 2x 

879SE-015F-FG 2x 

878KSE-019C-FG 2x 

878KSE-019F-FG 2x 

878KSE-022C-FG 2x 

878KSE-022F-FG 2x 

879KSE-021C-FG 2x 

879KSE-021F-FG 2x 

879KSE-024C-FG 2x 

879KSE-024F-FG 2x 

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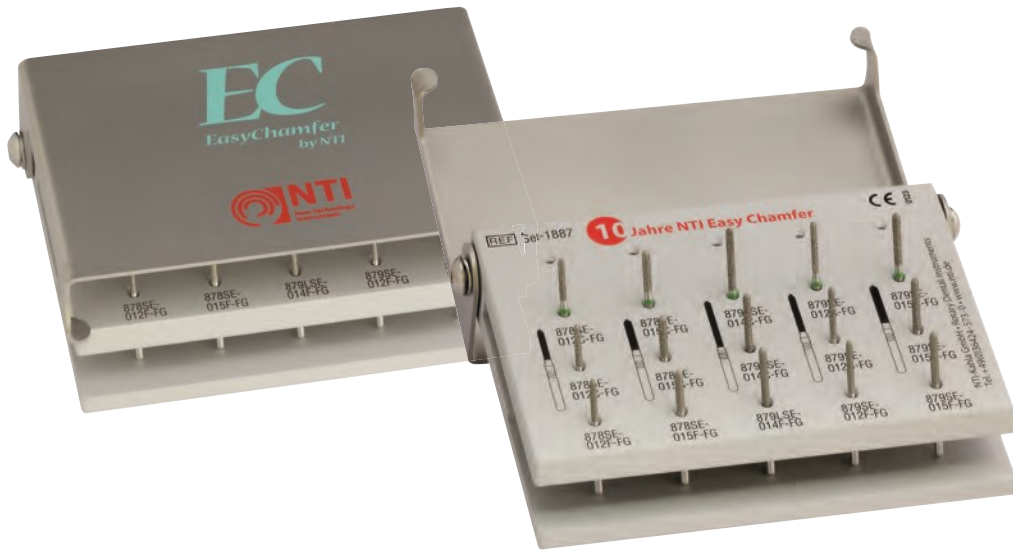
The abrasive-free tip facilitates contouring the cervical margin and produces an optimal deep chamfer, which meets the requirements for all-porcelain restorations.

The conically designed Torpedo has a 6° angle. This meets the recommendations of universities and the requirements of manufacturers of CAD/CAM scanner systems for perfect preparations.

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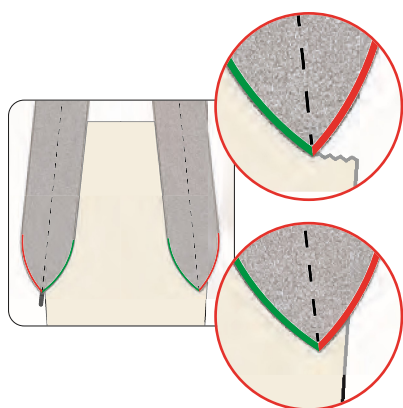
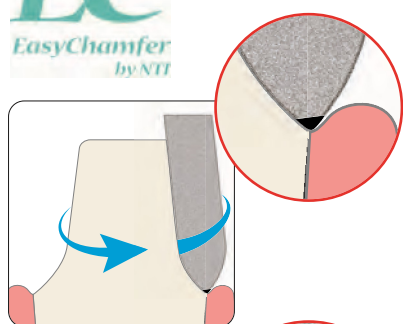
With fully diamonded preparation instruments, there is always the risk of creating so-called negative grooves or unevenness in the preparation as of the mid-section of the instruments.

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Instruments with a guide pin only permit a certain circular preparation depth.

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
The EasyChamfer with its partially diamonded instrument tips automatically precludes these risks. The instrument tip is actively involved in preparation.

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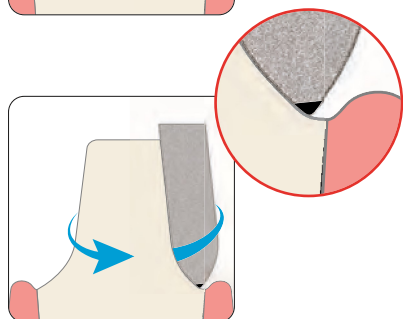
Within the preparation, the instrument supports itself cervically as the non-diamonded tip can run on the chamfer without damaging the chamfer. There is no abrasion at the tip of the instrument.

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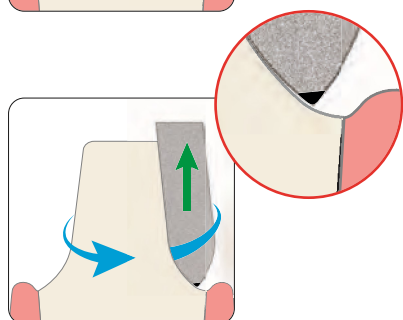
By intentionally raising the instrument during the preparation movement (blue arrow) in occlusal direction (green arrow), abrasion is activated and deeper shaping of the chamfer is always possible. Subsequently, the instrument tip is actively involved in the preparation again.

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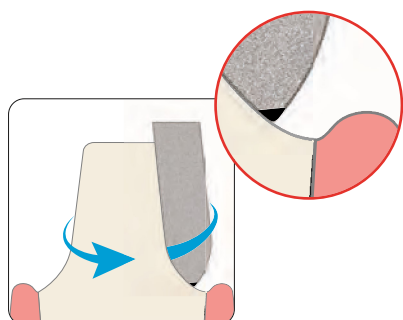By applying this technique, the preparation can be made as wide as desired with just a single instrument without damaging the gingiva. The position of the preparation margin can be changed further to gingival or subgingival at any time.

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