edenta

EDENTA DENTAL 2023/2024

ZAHNARZT

DENTIST

CABINET DENTAIRE

POLISHER
TUNGSTEN CARBIDE
STEEL
DIAMOND
ENDODONTICS
ROOT POSTS
ACCESSORIES

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All rights reserved.

Reproduction, also by extract and reproducing photos are only permitted with written authorization of EDENTA AG, Switzerland.

All offers, orders and deliveries are subject to Edenta's „General sales and Delivery Terms“. We reserve the right to modify our range of products and their design as well as to deviate from the illustrations and data shown.

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Quality Made in Switzerland

EDENTA supplies a coordinated instrumentarium of precision instruments for all types of treatment. The instruments are suitable for the latest preparation techniques and materials with fewer instrument changes and a shorter preparation time.

EDENTA instruments have a special cut and blade geometry to suit different materials, ensuring effective cutting and producing optimum results.

Comprehensive product range

On the basis of our research and development we can guarantee a product range that always provides state-of-the-art technology.

A comprehensive product range of rotary instruments for dentistry and dental technology from a one-stop supplier.

EDENTA online

Keep up to date with our homepage.
www.edenta.com

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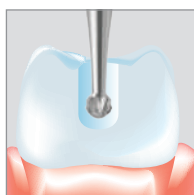
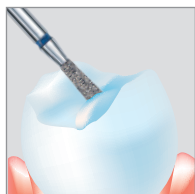
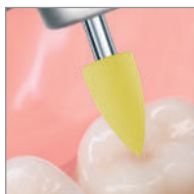
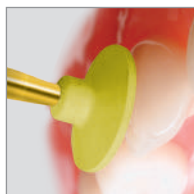
Head Master, aktiv, konisch	• Head Master, active, tapered	• Head Master, active, conique
Head Master, passiv, konisch	• Head Master, passive, tapered	• Head Master, passive, conique
Head Master, aktiv, zylindrisch	• Head Master, active, cylindrical	• Head Master, active, cylindrique
Head Master, passiv, zylindrisch	• Head Master, passive, cylindrical	• Head Master, passive, cylindrique
Pin Master, aktiv	• Pin Master, active	• Pin Master, active
Post Master, passiv	• Post Master, passive	• Post Master, passive
Retopin	• Retopin	• Retopin
Tri-Star	• Tri-Star	• Tri-Star
Tri-Jet	• Tri-Jet	• Tri-Jet

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Systematic finishing and polishing

Innovative polishers for all polishing stages, from contouring and finishing to polishing and high-lustre polishing.

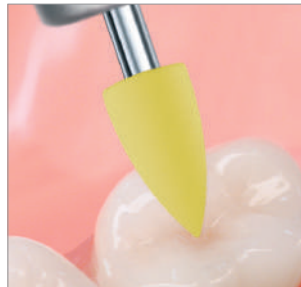
Polishers for improved quality and performance suitable for concave and convex surfaces, interdental and occlusal areas and for a natural lustre on all types of filling materials.

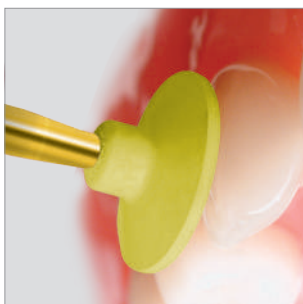
Konturierung, Ausarbeitung der anatomischen Form.
Contouring, finishing the anatomical shape.
Façonnage, ébauche de la forme anatomique.

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			Seite Page
Composite Polierer	Composite Polisher	Polissage de composite	08 - 11
DiaGloss	DiaGloss	DiaGloss	
CompGloss	CompGloss	CompGloss	
TopGloss	TopGloss	TopGloss	
Exa Special	Exa Special	Exa Special	
Exa Composite	Exa Composite	Exa Composite	
Flexi-Snap	Flexi-Snap	Flexi-Snap	
Keramik Polierer	Polishers for Ceramic	Polissage de céramique	12 - 15
CeraGloss	CeraGloss	CeraGloss	
StarGloss	StarGloss	StarGloss	
StarTec	StarTec	StarTec	
Cerapol plus	Cerapol plus	Cerapol plus	
Exa Cerapol	Exa Cerapol	Exa Cerapol	
Cerapol Super	Cerapol Super	Cerapol Super	
Klebstoffentferner	Adhesive Remover	Retrait des résidus adhésifs	14 - 15
OrthoFlex	OrthoFlex	OrthoFlex	
Polierer für Edelmetall-Legierungen	Precious Alloy Polishers	Polissage des métaux précieux	16 - 17
Alphaflex	Alphaflex	Alphaflex	
Alphaflex supergrün	Alphaflex supergreen	Alphaflex supervert	
Goldstar	Goldstar	Goldstar	
Polierer für Amalgam	Amalgam Polishers	Polissage des amalgames	18 - 19
Amalgam Reducer	Amalgam Reducer	Amalgam Reducer	
Nylon Bürste	Nylon Brush	Nylon brosse	
Universal Polierer	Universal Polishers	Polissage universel	18 - 19
Exa Intrapol	Exa Intrapol	Exa Intrapol	
Exa Dental	Exa Dental	Exa Dental	
Prophylaxe Polierer	Prophylaxis Polishers	Polissage prophylactique	20 - 25
Prophylaxe	Prophylaxis	Prophylaxie	
Prophylaxe Bürstchen	Prophylaxis brushlets	Brosses de prophylaxie	
TopBrush	TopBrush	TopBrush	
Polierer für Kunststoffe	Polisher for Acrylics	Polissage des matériaux acrylique	26 - 29
ExaTechnique grün/grau/gelb	ExaTechnique green/grey/yellow	ExaTechnique vert/gris/jaune	
Acrylic Polierer blau	Acrylic Polisher blue	Polissoir bleu pour acrylique	
Softcrack	Softcrack	Softcrack	
Hartmetallfräser - 53AC, 75AC, 75, 10, 20	TC Cutter - 53AC, 75AC, 75, 10, 20	Fraises en carbure de tungstène - 53AC, 75AC, 75, 10, 20	
Prep Sets	Prep Sets	Prep Sets	30 - 36
DiaGloss Composite Kit	DiaGloss Composite Kit	DiaGloss Composite Kit	
Composite Finishing Kit	Composite Finishing Kit	Composite Finishing Kit	
CeraGloss Ceramic Kit	CeraGloss Ceramic Kit	CeraGloss Ceramic Kit	
Porcelain Veneer Kit	Porcelain Veneer Kit	Porcelain Veneer Kit	
Amalgam Veneer Kit	Amalgam Veneer Kit	Amalgam Veneer Kit	
Amalgam Polishing Kit	Amalgam Polishing Kit	Kit de polissage d'amalgame	
Prophylaxis Kit (Pasteless)	Prophylaxis Kit (Pasteless)	Prophylaxis Kit (Pasteless)	
Acrylic Polisher blue Kit	Acrylic Polisher blue Kit	Acrylic Polisher blue Kit	
Orthodontic Burs Starter Kit	Orthodontic Burs Starter Kit	Orthodontic Burs Starter Kit	
Denture Adjustment Kit	Denture Adjustment Kit	Denture Adjustment Kit	
CeraGloss Kit	CeraGloss Kit	CeraGloss Kit	
TopGloss Kit	TopGloss Kit	TopGloss Kit	
Ceramic Finishing Kit	Ceramic Finishing Kit	Ceramic Finishing Kit	
Composite Finishing Kit	Composite Finishing Kit	Composite Finishing Kit	
Keramische Schleifkörper	Ceramic Abrasives	Abrasifs à liant céramique	37
Schleifkörper	Abrasives	Abrasifs	
Arkansas	Arkansas	Arkansas	

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- 1. Step:** FINISHER (pink) - abrasive polishing
2. Step: POLISHER (white) - high-gloss polishing

- Etape 1:** FINITION (rose) - polissage abrasif
Etape 2: POLISSOIR (blanc) - brillantage

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Two-step polishing system specially developed for producing an intraoral high-lustre polish on all composite filling materials.

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An innovative polishing matrix with diamond particles guarantees a high lustre in minimum time and a high service life of the polisher.

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1039RA & 1139RA - The highly flexible polishing lamellae adapt to any surface structure thus creating perfect polishing results, even on areas difficult to polish such as occlusal surfaces.

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Application - Use coolant, 50mL/min., when polishing to avoid heat build-up.

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External cooling should be used when polishing with lamellae polishers, which should be directed between the polisher and composite filling to avoid excess spray water in the direction of the handpiece and optimally cool the area being polished.

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This also increases the polishing efficiency, as the fine composite particles, which are removed during polishing, are rinsed off with the water. This results in a perfect high-lustre.

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Diamond-impregnated micro-polisher for high-gloss polishing of all composite and compomere fillings.

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- Easy to use.
- High gloss result in only one step.

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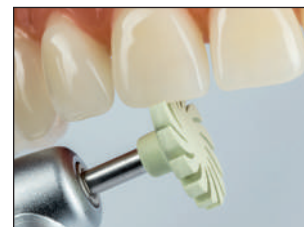
The ideal polisher for all micro-filler composites. Produces a smooth, durable surface polish.

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The universal macrofiller composite polisher. Produces a perfect finish without damaging the enamel.

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Also suitable for removing cement residue from orthodontic bands (brackets).

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Cups and discs for polishing subgingival and interproximal surfaces.

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Flames and points for polishing occlusal and lingual surfaces.

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Flexi-Snap Polishing discs produce less surface roughness than conventional discs, which results in a higher lustre.

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Polishing disks for creating a natural highlustre on all composite materials.

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For contouring and finishing as well as prepolishing and high-lustre polishing.

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These thin, highly flexible polishing disks with diameters of 12 and 8 mm can be used perfectly, even in interdental spaces. The grit sizes and functions are denoted by colour codes.

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These disks are coated on both sides for working quickly without needing to replace the disk or turn it over.

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As the Flexi-Snap polishing disks do not have metal hubs, they do not scratch or discolour the surface of the teeth or filling.

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Surface roughness [µm]

Depending on the grit size, Flexi-Snap polishing disks reduce the surface of the restorative more than conventional disks, thus leaving less surface roughness and enhancing the lustre.

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The surfaces of the tooth and filling are polished gently.

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3 - step diamond polishing system suitable for all porcelain fillings, partial and full-crown restorations.

Polishers with natural diamond grit suitable for finishing, polishing and high-lustre polishing of porcelain surfaces.

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3039RA & 30039RA - The highly flexible polishing lamellae adapt to any surface structure thus creating perfect polishing results, even on areas difficult to polish such as occlusal surfaces.

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- Step 1 = Green:** Finishing, for rapid material reduction.
- Step 2 = Blue:** Polishing, for smoothing the surface
- Step 3 = Yellow:** High-lustre polishing, without polishing paste

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3 - step diamond polishing system suitable for all porcelain fillings, partial and full-crown restorations. Polishers with natural diamond grit suitable for finishing, polishing and high-lustre polishing of porcelain surfaces. A second glaze firing is not necessary following adjustment.

R2830RA & R2840RA - The highly flexible polishing lamellae adapt to any surface structure thus creating perfect polishing results, even on areas difficult to polish such as occlusal surfaces.

The polishing lamellae prepare the material surface very gently, preserving the original surface structure of the restoration.

- Step 1 = BLUE:** Finishing, for rapid material reduction.
- Step 2 = PINK:** Polishing, for smoothing the surface
- Step 3 = GREY:** High-lustre polishing, without polishing paste

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2-step diamond polishing system

For reliable and gentle preparation of zirconium dioxide, lithium disilicate, zircon-reinforced lithium silicate (ZLS), hybrid-ceramic and all commonly used veneering porcelains.

After milling ceramic restorations have a surface roughness. To reduce the abrasion on the opposing tooth the surface roughness must be reduced.

- Step 1 = Purple:** **Medium grit**
For smoothing the surfaces and preparation for final glaze.
- Step 2 = Yellow:** **Superfine grit**
For high-shine polishing without polishing paste

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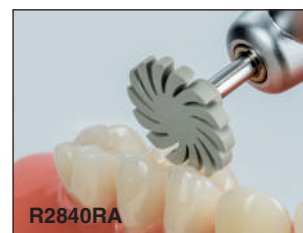
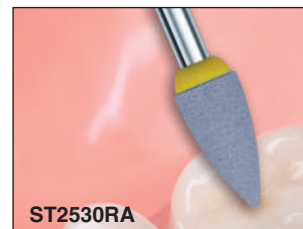
1 - step diamond polishing system suitable for all porcelain fillings, partial and full-crown restorations.

Polishers with natural diamond grit for polishing porcelain surfaces to a high-lustre. A second glaze firing is not necessary following adjustment.

Easy to use and saves time with optimum polishing capacity.

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2 - step finishing and polishing system suitable for all porcelain fillings, partial and full-crown restorations.

Polishers for finishing and polishing porcelain surfaces. A second glaze firing is not necessary following adjustment.

For porcelain/ceramics **grey-white**, abrasive, eliminates scratches and smoothens the surface.

For porcelain/ceramics **pink**, provides a gentle finish, retains anatomic structure.

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Polishers for all porcelain fillings, partial and full-crown restorations.

For polishing porcelain surfaces to a high-lustre, producing a natural sheen without the use of polishing paste.

A second glaze firing is not necessary following adjustment.

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Used for reliable, gentle removal of residual adhesive without damaging the natural tooth structure achieved by a special grit, which despite having a high polishing capacity is not harder than the natural tooth structure.

Especially reliable alternative to adhesive or cement residue removal using rotary diamond or metal instruments.

Speed opt. 15,000 rpm.

After removal of the adhesive the teeth are re-polished using: Exa Cerapol polisher.

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2 - step finishing and polishing system suitable for gold, amalgam and composite. Polishers impregnated with high-grade polishing agents for rapid finishing without damaging the enamel.

Step 1 = Brown: Prepolishing (finishing).

Instructions for use:

- Apply only light pressure when polishing
- Adhere to the recommended rpm
- Polishing paste is not required
- Use coolant spray when polishing

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SuperGreen: High-lustre polishing. High-lustre polishers suitable for gold, amalgam and composite. Polishers impregnated with high-grade polishing agents for high-lustre polishing without damaging the enamel.

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3 - step polishing system suitable for gold and precious metal alloys. Polishers with special polishing agents for finishing, polishing and high-lustre polishing with an antioxidant effect. Produces natural intraoral contours and aesthetics.

Step 1 = Green: Finishing, removes scratches, smoothes

Step 2 = Yellow: Prepolishing with slight lustre

Step 3 = Pink: High-lustre polishing with antioxidant effect

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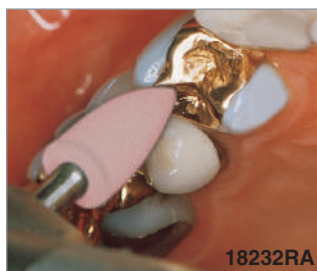
Polishers impregnated with high-grade polishing agents suitable for rapid initial reduction and finishing of amalgam fillings.

Can be taken over the tooth enamel when polishing without damaging the enamel. This prevents the formation of marginal gaps.

Polishing and high-lustre polishing is completed with brown and green Alphaflex polishers.

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Made from nylon bristles and suitable for polishing amalgam and precious metal alloys with polishing paste.

Ideal for polishing occlusal and approximal surfaces.

Instructions for use:

- Apply only light pressure when polishing
- Adhere to the recommended rpm
- Always use polishing paste

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Universal polisher, suitable for composites. Produces a smooth, durable surface lustre.

Polishers - shapes:

Cups – Polishing convex surfaces and fissures, ideal for polishing interdental spaces due to fine edges.

Flames/Points – Polishing concave surfaces, ideal for polishing the occlusal surface.

Discs – Polishing convex surfaces, ideal for polishing smooth surfaces of anterior teeth.

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Universal polisher, suitable for all alloys.

Produces a durable lustre. Ideal for polishing the junction between the porcelain and metal.

Polishing cup polish subgingival and interproximal surfaces safely.

Flame and point polishers are suitable for polishing occlusal and lingual surfaces.

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Pasteless prophylaxis. Optimal cleaning and polishing of the tooth surface with minimal abrasion.

The newly developed prophylactic polishers with optimal cleaning and polishing properties protect the tooth structure thanks to a special polishing matrix with unique abrasive properties and low friction coefficients. This greatly reduces any unpleasant heat build-up in the tooth for the patient.

Pasteless cleaning and polishing ensures a clear view of the tooth being treated, which avoids damaging the gingiva. This eliminates any unhygienic spray removal of polishing paste and saliva, which ensures consistent, efficient cleaning and polishing.

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Polisher with contra-angle shank and spray guard, without abrasive media for removing dental plaque, with polishing paste.

The integrated spray guard prevents the polishing paste getting into the contra angle.

Optimal motor speed = 1,500 rpm

Colour coding indicates the different hardness of the polishers:

- White = standard polisher, for removing dental plaque and polishing
- Pink = soft polisher for removing dental plaque in the region of the sulcus, to protect the soft tissue
- Yellow = hard polisher for removing heavy dental plaque and discoloration

Single-use, non-autoclavable

Polishers with screw thread and spray guard, without abrasive media for removing dental plaque, with polishing paste.

For screw retention in all known prophylaxis contra-angles or in the reusable screw mandrel 4039RA.

Optimal motor speed = 1,500 rpm

Colour coding see under Prophylaxis Cups (RA)

Single-use, non-autoclavable

Snap-on polishers with spray guard, without abrasive media for removing dental plaque, with polishing paste.

Used in combination with the reusable snap-on mandrel 4037RA

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Pasteless prophylaxis

Only one instrument is required for cleaning and polishing in one working stage. Maximum cleaning and polishing performance without polishing paste ensures minimal abrasion of tooth enamel and dentine. Time-saving cleaning and polishing because repeated uptake of polishing paste is no longer required. Optimal motor speed 5'000 rpm, with spray and light pressure application, reusable, autoclavable.

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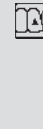
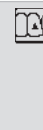
Nylon brushes with a contra-angle shank for removing dental plaque, with polishing paste on occlusal surfaces, cusps and cusp slopes.

The flexible nylon brushes also allow cleaning beneath the gingival margin.

Optimal motor speed = 1,500 rpm

Colour coding indicates the different hardness of the nylon bristles:

- White = standard brushes, for removing dental plaque and polishing
- Pink = soft brushes for removing dental plaque in the region of the sulcus, to protect the soft tissue
- Yellow = hard brushes for removing heavy dental plaque and discoloration

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Nylon brushes with screw thread for removing dental plaque, with polishing paste.

For screw retention in all known prophylactic contra-angles or in the reusable screw mandrel 4039RA.

Large cup: for large surface polishing in the anterior and posterior regions.

Small cup: or polishing anterior and cervical regions as well as the occlusal morphology in the posterior region.

Optimal motor speed = 1,500 rpm
Colour coding see under Prophylaxis Brushlets

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Screw-in mandrel with contra-angle shank for all screw-in polishers and screw-in brushes.

autoclavable, reusable

Snap-on mandrel with contra-angle shank for all snap-on polishers and snap-on brushes.

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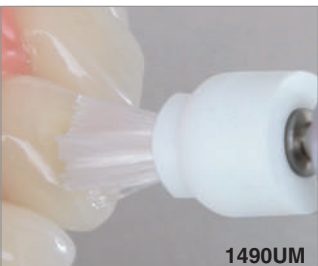
Polishers with plastic contra-angle shank and spray guard, prevents penetration of polishing paste into the contra-angle. For single use with polishing paste, non-autoclavable.

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Polishers with plastic contra-angle shank and spray guard. Special nubs on the outside and inner diagonal lamellae constantly guide the paste to the edge of the cup and thus always towards the teeth, ensuring optimum tooth cleaning while simultaneously reducing the frictional heat. For single use, non-autoclavable.

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Nylon brushes with plastic contra-angle shank for removing accretion, with polishing paste. For single use, non-autoclavable.

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Prophylaxis treatment, maximum cleaning and polishing of the tooth surface

Healthy teeth and gingiva – with the correct prophylaxis. The new prophylaxis polishers with improved polishing properties smooth tooth surfaces to prevent renewed accumulation of bacteria. The polishers are gentle on the tooth structure during polishing and heating of the teeth, which is unpleasant for the patient, is greatly reduced. This guarantees thorough, reliable cleaning of the teeth. Colour coding identifies the different hardness of the polishers: with plastic shank: blue = standard polisher for removing accretion and polishing, pink = soft polisher for removing accretion near the sulcus, for protecting the soft tissue. For single use with polishing paste, non-autoclavable.

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TopBrush is a polishing system for use without polishing paste. Each bristle is an individual polishing instrument.

Functional principle

Nylon brushes with integrated polishing particles made from silicone carbide for the removal of heavy dental plaque without the use of polishing paste as well as for high-lustre polishing of composites, compomers, porcelain and glass ionomer cement.

One-step polishing system for pre-polishing and high-lustre polishing using only one instrument.

Use without polishing paste.

The special fibres contain silicone carbide (SiC) as the abrasive medium. As the bristles become worn fresh SiC is constantly exposed on the surface of the fibre. This produces consistently high polishing performance.

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3 shapes of brushes for preparing all occlusal and proximal surfaces.

Large cup

For polishing large surfaces in the posterior region as well as removing discoloration.

Small cup

For polishing anterior, cervical and occlusal restorations as well as removing gingival discoloration.

Point

For polishing occlusal fissures and also in the proximal region.

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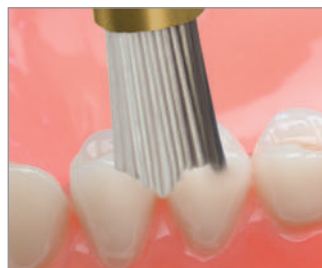
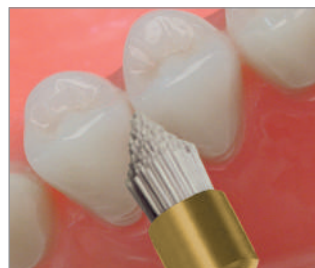
Optimal motor speed = 3,000 rpm

Use without polishing paste

Only apply light pressure during preparation

Avoid heat build-up

Autoclavable and can be reused several times

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Exa Technique green

3 - step polishing system suitable for denture and orthodontic acrylics.

Pretrimming, for rapid material reduction.

Step 1 = Green: Coarse grit

Silicone polisher suitable for finishing and contouring. Removes scratches and smoothes the material surface.

Also suitable for adjustment of permanently soft acrylics.

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Exa Technique grey

3-step polishing system suitable for denture and orthodontic acrylics.

Polishing, for smoothing surfaces.

Step 2 = Grey: Medium grit

Silicone polisher for smoothing and condensing material surfaces. Produces a slight lustre.

Suitable for preparing for high-lustre polishing.

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Exa Technique yellow

3-step polishing system suitable for denture and orthodontic acrylics.

High-lustre polishing.

Step 3 = Yellow: Fine grit

Silicone polisher for optimally smoothing and condensing material surfaces.

Produces a high-lustre polish without polishing paste.

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Acrylic Polisher blue

2-step polishing system suitable for denture acrylics.

Adjustments to the palatal and lingual aspects of the denture are completed quickly without using tungsten carbide cutters. Polishers with maximum service life.

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High-lustre polishing is completed by using yellow Exa Technique polishers.

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Soft, elastic Multi-Layer grinding discs with optimal grit size for rapid and effective operation on soft and elastic resins, for grinding and contouring of soft relinings, positioners, flexible splints, mouth guards, wound dressings.

- High material reduction, no smearing
- Low heat build-up
- Smooth surfaces without streak formation
- Ergonomic shape

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53AC

Cutter with AC-Coating

Used for trimming denture base acrylics. The fine-coarse cut is specially tailored to trimming in acrylic work, is easy to use, does not catch and produces smooth material surfaces.

75AC

Cutter with AC-Coating

For trimming of soft acrylics / soft relinings.

- High material reduction, no smearing
- Low heat build-up
- Smooth surfaces

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TC Cutter - cut 75.

Plain toothing with transverse.

For trimming of soft acrylics/soft relinings.

- High material reduction, no smearing
- Low heat build-up
- Smooth surfaces without streak formation
- Ergonomic shape

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10

TC Cutter - cut 10.

Suitable for all dental materials. For coarse abrasion and optimal surface quality.

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TC Cutter - cut 20.

Suitable for all dental materials.

It smoothens the surface and it enables accurate operation on any structure.

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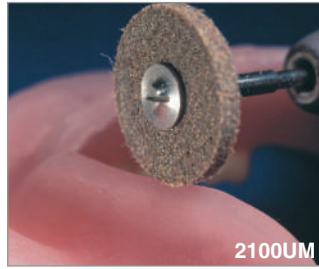
Cutter with AC-Coating

Instruments with ZrN (zirconium nitride) hard coating. For smoother operation and reduced heat generation due to less clogging. For fast cutting without clogging.

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Unique diamond-impregnated surface treatment system for contouring and high-gloss polishing of composites.

Pink finisher ensures a very fast pre-polished surface.

With the white polisher one will achieve a quick and easy high-shine. No dust, no smearing.

The set contains 6 finishing & 6 polishing instruments.
Application: Use coolant spray when polishing.

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Kit consists of 11 assorted instruments: 2 white Arkansas Abrasives in FG Shank for removing subgingival flash.

3 superfine diamonds (yellow ring) in FG Shank for finishing off composite fillings.

6 special silicone points in RA Shank for polishing all types of composite materials.

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Set enthält 6 CeraGloss RA Polierer.

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Unique diamond-impregnated surface treatment system for high-gloss polishing of surgery ceramics. Time-saving, convenient single step process.

Natural, high temperature-resistant rubber with top-grade natural diamond particles which ensure rapid surface reduction combined with a high-gloss result of unsurpassed excellence.

Set contains 6 CeraGloss RA polishers.

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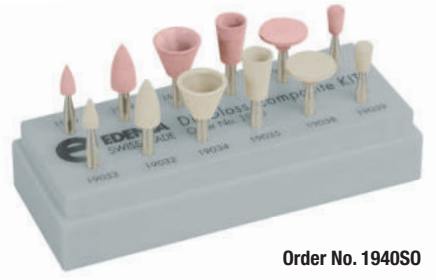
Kit consists of 9 assorted instruments in RA and FG shank, specifically designed for the Porcelain Veneer Technique. The fine diamond (red ring) is used in preparation of the tooth and the special silicone CeraPol Plus diamond impregnated points are used to obtain a smooth and satin finish.

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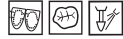
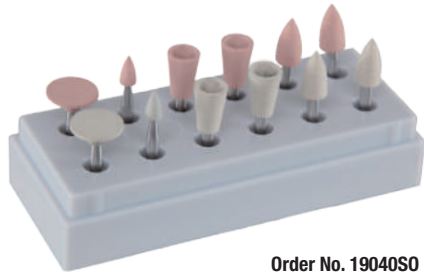
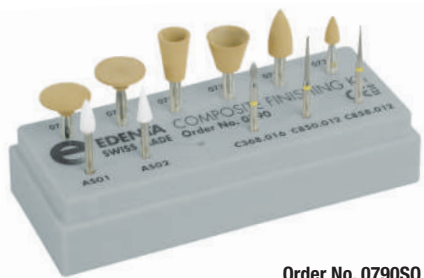
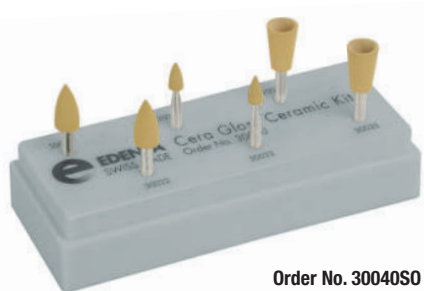
Kit consists of 6 TC FG Finishing Burs for bulk abrasion of amalgam and to work out fissures, and 6 Amalgam Reducers for fine reduction and finishing of amalgam fillings.

An excellent marginal adaptation is thus ensured.

Its application can be effected dry or by using water-spray.

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Kit consists of 12 assorted instruments in cup and bullet shape, with RA shanks.

PROCEDURE: **Step 1** = start with black Amalgam Reducer. These are designed for substance reduction and shaping of amalgam alloys after the carving procedure.

Step 2 = Now use the brown Alphaflex. These are used to smoothen the restoration to a satin finish.

Step 3 = Using the green Alphaflex. These will polish the surface to a super high shine. Its application can be effected dry or by using water-spray.

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Specially designed polishers which incorporate a superfine abrasive grit, for tooth cleaning without the use of polishing paste.

Optimum cleaning and polishing of the tooth surface with minimum abrasion.

Minimum abrasion on the enamel and dentine.

Gentle on the tooth structure.

Negligible loss of tooth structure.

Application: Use coolant spray when polishing. Without polishing paste

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New type of long-chain silicones with specific abrasive bodies which ensure controlled & rapid reduction combined with a smooth prepolish.

Adjustments to the palatal and lingual aspects of the denture are completed quickly without using tungsten carbide cutters.

Step 1 = Dark blue: Course grit for pretrimming.

Step 2 = Light blue: Medium grit for smoothing.

High-lustre polish: Yellow Exa Technique polishers.

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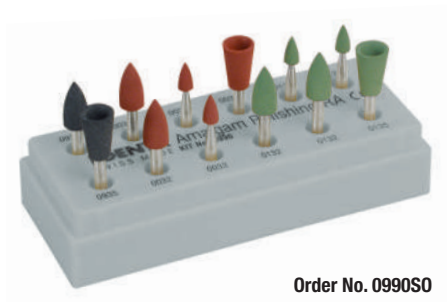
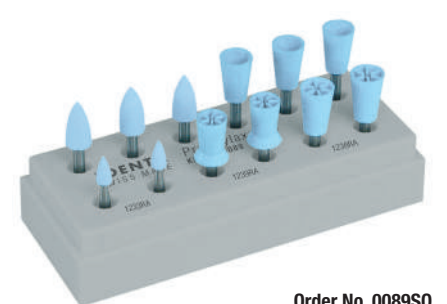
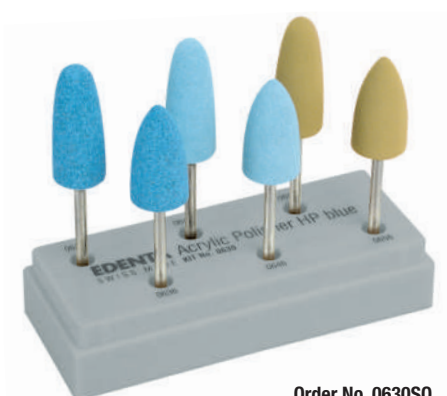
The set contains 4 different trimmers and is utilized mainly for adjustments in the Laboratory. All 4 burs/cutters are generally used to trim acrylics of functional appliances, and in the case of the jumbo acrylic cutter to trim the bulk stone off working and study models.

The two grey medium-grit polishers are used for rough finish on acrylic repairs and the yellow fine-grit polishers for high-shine.

The polishing wheel and mandrel are primarily used to polish wires and bands that have been soldered.

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Kit consists of 6 assorted points in HP Shank. These points are used specifically for chair-side adjustments and polishing adjusted surfaces for all resin prostheses.

1 TC Trimmer for cutting, 4 silicone points ExaTechnique for initial polishing, 1 silicone point for high lustre final polishing.

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3-step diamond polishing system suitable for all porcelain fillings, partial and full-crown restorations. Polishers with natural diamond grit suitable for finishing, polishing and high-lustre polishing of porcelain surfaces. A second glaze firing is not necessary following adjustment.

- Step 1 = Green: Finishing, for rapid material reduction.
- Step 2 = Blue: Polishing, for smoothing the surface
- Step 3 = Yellow: High-lustre polishing

Application: Use coolant spray when polishing.

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Diamond micropolishers – superior quality surface lustre on all composite and compomer fillings.

- Easy to use.
- High-lustre polish in only one step.

Instructions for use:

- Only apply light pressure when polishing
- Adhere to the recommended rpm
- Polishing paste is not required
- Use coolant spray when polishing

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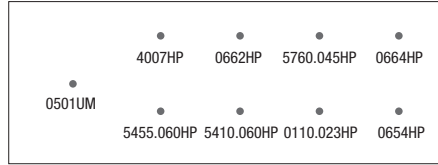
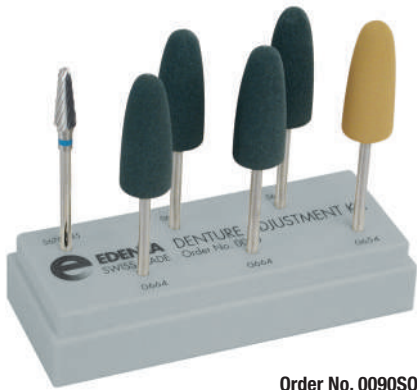
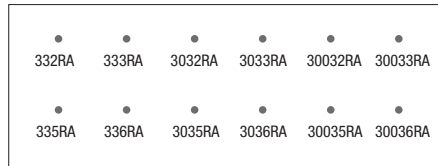
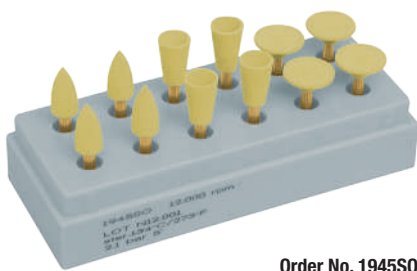
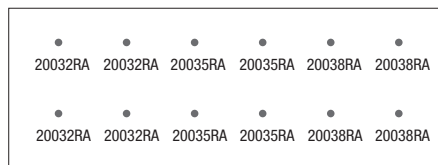
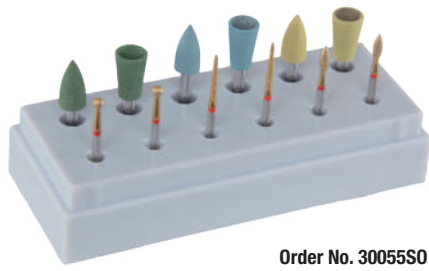
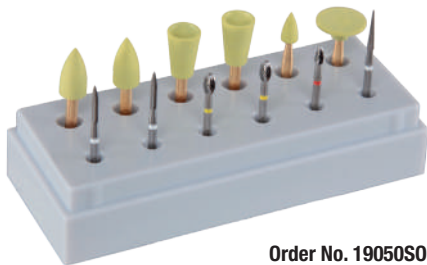
6 DiaCut diamond rotary instruments and 6 CeraGloss diamond polishers for adjusting zirconia all-ceramic materials in the dental practice.

DiaCut fine-grit diamonds (red ring) are used for adjusting occlusal contact points. Polishing is completed using CeraGloss polishers. Use cool spray when polishing.

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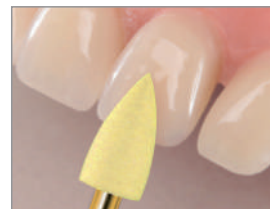
After the composite has been polymerized, excess is removed and contours are prepared with yellow-ring tungsten carbide finishers. The composite is then smoothed and polished to a high lustre using TopGloss diamond-grit polishers.

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High-grade, silicon-carbide abrasive stones with ceramic binder.

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Ideal for rapid, accurate contouring and finishing of porcelain, acrylic and metal alloys (gold, silver, amalgam).

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Arkansas

FG 15.000
RA 7.000
12/100

L mm	6,0	7,0	6,0	6,0	3,0	030
Size $\varnothing 1/10$ mm	025	025	025	025	030	030
Order No.	AS 01.025 FG	AS 02.025 FG	AS 03.025 FG	AS 04.025 FG	AS 05.030 FG	AS 06.030 FG
ISO No. 635 314...	243 505 025	161 505 025	171 505 025	110 505 025	013 505 030	001 505 030
Order No.	AS 11.025 RA	AS 12.025 RA	AS 13.025 RA	AS 14.025 RA	AS 15.030 RA	AS 16.030 RA
ISO No. 635 204...	243 505 025	161 505 025	171 505 025	110 505 025	013 505 030	001 505 030

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High-grade, fine-grained aluminium oxide abrasive stones with ceramic bond.

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Ideal for rapid, accurate finishing (very fine preparation) of composite materials. Finishing enamel, prepared teeth, composites and porcelain.

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Guarantee for vibration-free preparation.

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Tungsten carbide instruments for precise preparations

Every preparation is an integrated procedure. With the Edenta range of instruments for preparations the procedure flows smoothly from one working stage to the next, with fewer instrument changes and a shorter preparation time.

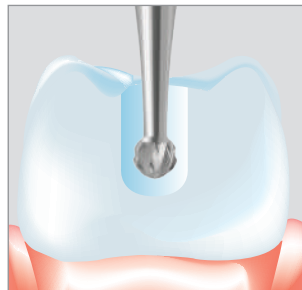
The Edenta tungsten carbide range provides universal precision instruments suitable for many applications, including cavity preparations, crown separation and periodontal treatment.

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Exkavation, Retention, Perforation, Penetration, Trepanation.
Excavation, retention, perforation, penetration, trepanation.
Excavation, rétention, perforation Pénétration, trépanation.

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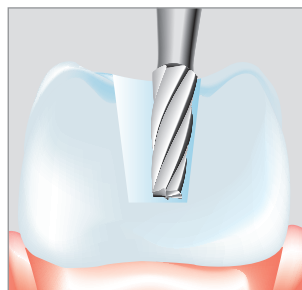
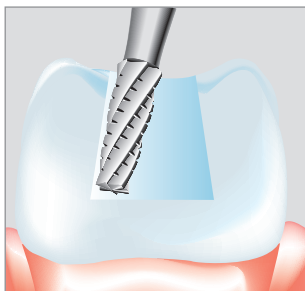
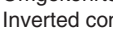
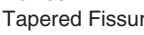
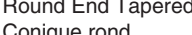
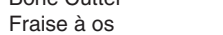
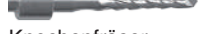
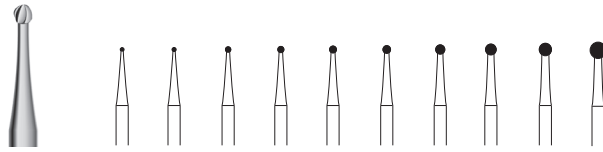
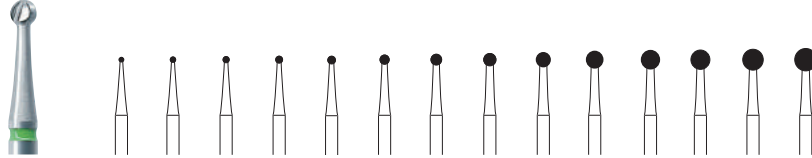
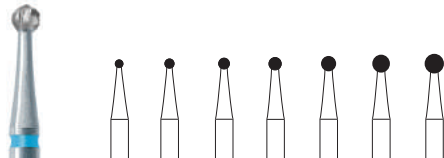
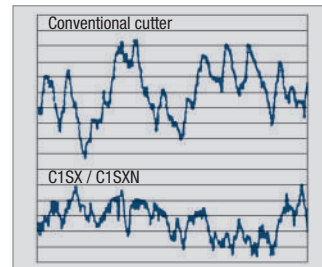
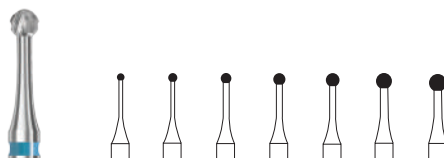
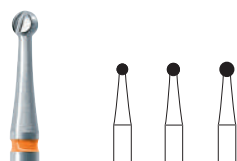
Minimally invasive excavation using the C1SDR tungsten carbide bur, removes only soft, carious dentine.

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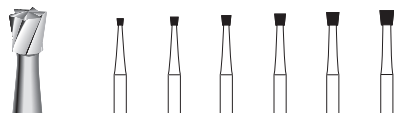
The new C1SDR tungsten carbide bur is impressive due to its cutting efficiency in carious, soft dentine while it hardly removes any healthy, hard dentine. This is the perfect combination for minimally invasive excavation.

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Adhesive Remover

For removal of residual resin after debonding of orthodontic brackets. Tungsten carbide burs at low speeds are most efficient for gross removal of the residual resin and produced the finest scratch pattern with the least enamel loss. Advantages: a special TiN coating smoothes the tungsten carbide surface of the adhesive remover, preventing clogging of the instrument blades during removal of the adhesive. This enables quiet, controlled removal of the residual adhesive.

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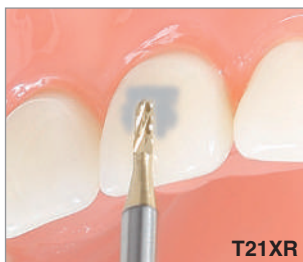
Reduction of adhesive residue with the adhesive remover.

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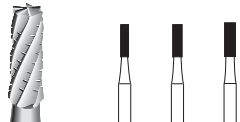
Caution: Suitable for all types of metal alloys, titanium and low-fusing porcelain and composite facings.

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Crown Cutter C35L for metal crowns

Specially coordinated for the separation of non-precious metal and gold crowns with an improved cutting capacity and service life compared with conventional crown cutters. The special compact blade configuration provides the C35L with controlled cutting behaviour and high cutting efficiency due to the optimally adapted tip-transversing blade with cross-cut. Use in the red contra-angle at 160,000 rpm; can also be used in a turbine.

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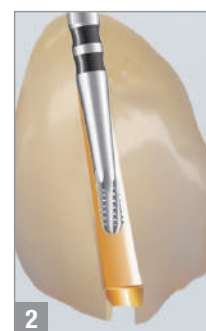
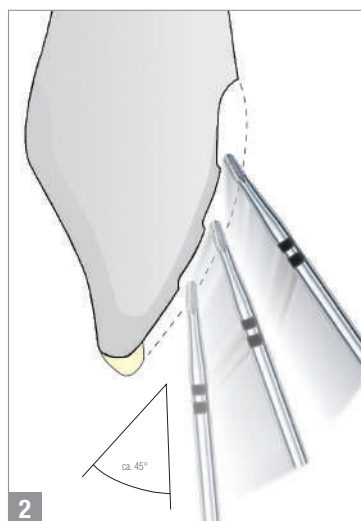
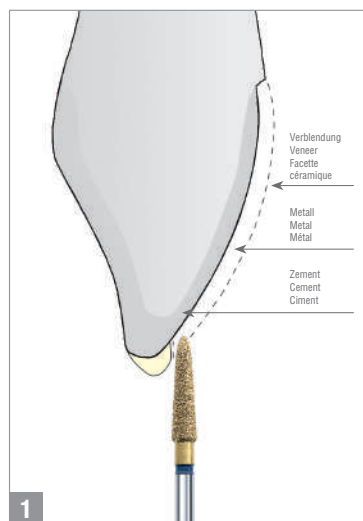
User recommendations:

When cutting crowns and bridges the following should be observed:

- The veneer** should first be separated with a diamond instrument.
- Crown material** reduction of the crown material layer, which means starting the procedure from top to bottom end several times. ⌚ max. 120'000 - 160'000 rpm
Contact pressure: 0.5 N

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Removal of all-ceramics restorations

To remove an all-porcelain restoration, cut the restoration open along the length of the axial wall to the centre of the occlusal surface or incisal edge. Then bend the restoration open with a suitable instrument to make it fracture.

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Crown Cutter for all-ceramic / zirconia

The high stability of the special grit produces an outstanding service life of the K-Diamond instruments. The new electroplating bonding prevents the diamond grit breaking out prematurely. This creates an unrivalled abrasive capacity, perfectly, quickly and reliably.

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Removal of all-ceramics restorations

To remove an all-porcelain restoration, cut the restoration open along the length of the axial wall to the centre of the occlusal surface or incisal edge using the sinter diamond disc order no.5114RA. Then bend the restoration open with a suitable instrument to make it fracture. No. 9920 Cleaning stone, (S. 134)

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2 Piece - TC Burs

Working part made out of durable HIP tungsten carbide, stainless steel bur shank, without nickel plating, ensure the instruments are gripped securely in the chuck.

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H1
Rund • Round • Rond
↻_{max.} 300.000
🗑️ 5

Shank Size	L mm	ISO	Order No.	US No.	FG
∅ ₁₀ mm					

H2
Umgekehrter Kegel • Inverted Cone • Cône renversé
↻_{max.} 300.000
🗑️ 5

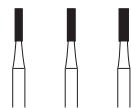
Shank Size	L mm	ISO	Order No.	US No.	FG
∅ ₁₀ mm					

H7
Birne • Pear • Poire
↻_{max.} 300.000
🗑️ 5

Shank Size	L mm	ISO	Order No.	US No.	FG
∅ ₁₀ mm					

H21
Zylinder • Cylinder • Cylindre
↻_{max.} 300.000
🗑️ 5

Shank Size	L mm	ISO	Order No.	US No.	FG
∅ ₁₀ mm					

H21R
Zylinder rund • Cylinder round • Cylindre rond
↻_{max.} 300.000
🗑️ 5

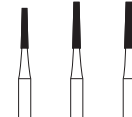
Shank Size	L mm	ISO	Order No.	US No.	FG
∅ ₁₀ mm					

H23
Konisch • Tapered Fissure • Conique
↻_{max.} 300.000
🗑️ 5

Shank Size	L mm	ISO	Order No.	US No.	FG
∅ ₁₀ mm					

H23L
Konisch lang • Long Tapered Fissure • Conique long
↻_{max.} 300.000
🗑️ 5

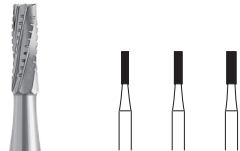
Shank Size	L mm	ISO	Order No.	US No.	FG
∅ ₁₀ mm					

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PowerCut

Maximum cutting performance without jamming or catching

Further development of the cutting blades with a reinforced cross-cut has optimized the cutting performance and quality of the PowerCut tungsten carbide cutter for all applications.

New blade design ensures more efficient, quicker preparation

The high speed and quality of the new PowerCut are impressive when preparing cavities, particularly in the case of molars. Its outstanding efficiency is also highlighted when removing amalgam and composite fillings. The high cutting performance and reinforced cross-cut ensure that any crown material can be cut quickly and effectively. The PowerCut is not however intended for preparing facing porcelain or all-ceramic. The blade geometry also makes it suitable for preparing titanium abutments.

PowerCut Crown Cutter - H31RPC

Separation of the Crown- & Bridge metal frame as well as the preparation of titanium abutments.

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PowerCut TC Burs

Removal of amalgam, composite fillings and preparation of titanium abutments.

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Instructions for use

PowerCut instruments can be operated in a turbine or micromotor with a red-ring contra-angle at a motor speed of 60'000 – 160'000 rpm. We recommend using a micromotor at 60,000 rpm for non-precious metal alloys. Porcelain facings should be removed with a diamond rotary instrument prior to using the PowerCut as a crown cutter.

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FG tungsten carbide burs for easy removal of amalgam and composite. Special surface conditioning guarantees optimum material removal, preventing the instruments becoming smeared.

Large sections of fillings are removed quickly without temperature build-up.

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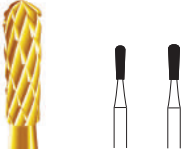
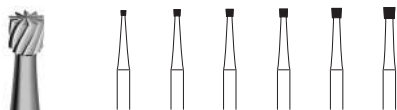
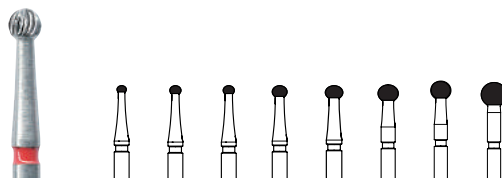
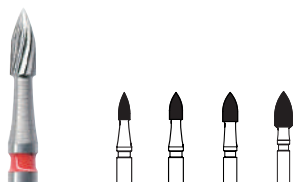
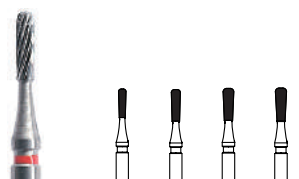
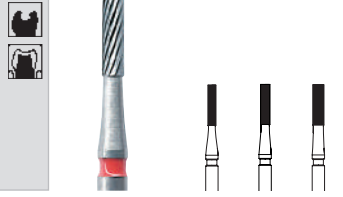
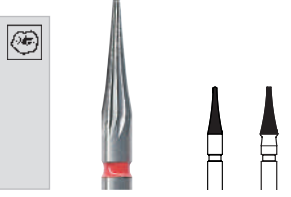
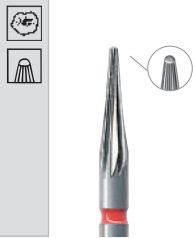
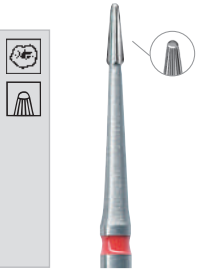
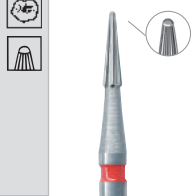
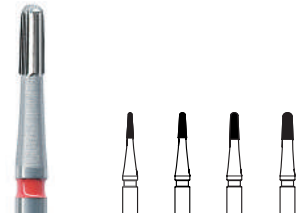
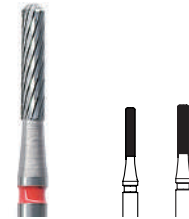
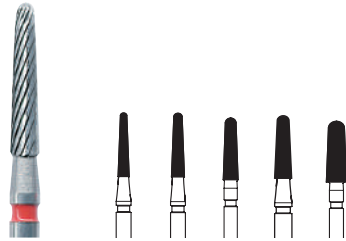
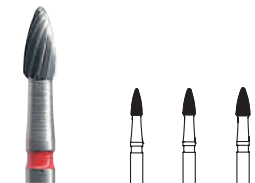
Intraoral trimming of titanium abutments
 The tungsten carbide instruments with a coarse cross-cut, which were specially developed for trimming titanium alloys, guarantee optimal contouring of titanium materials.

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The titanium surfaces are then finished using the congruently shaped red ring finishers.

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S-Finisher

Finishing a composite filling is reduced from 3 to 2 steps using the S Finishers.

Step (1) = S Finisher, step (2) = Ultra-fine Finisher (white ring). Highlustre polishing is then completed using the TopGloss one-step diamond grit polisher (3).

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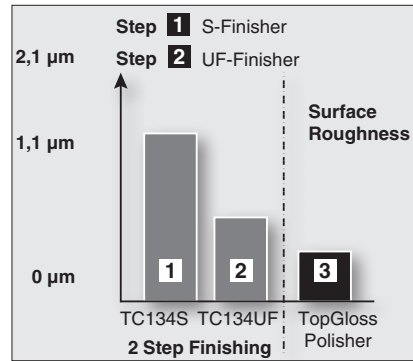
Finisher FGXL

Used for gentle preparation of lithium-disilicate glass ceramic (LS2) materials in the dental lab or in the dental practice for corrections on lithium-disilicate restorations. Blades with hard carbon coating (BHC). Use with spray cooling.

Attention: The tungsten carbide finishers are only approved for preparing lithium-disilicate glass ceramic (LS2) with a maximum flexural strength of 530 MPa.

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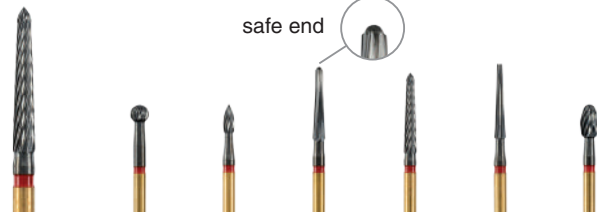
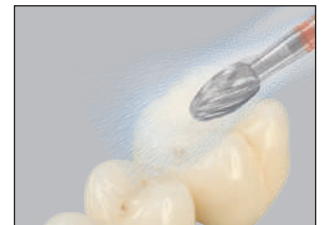
The specially developed tungsten carbide instruments provide a real alternative to curette preparation. The instruments enable root smoothing while preserving the tooth structure with visibly less groove formation than when using a curette.

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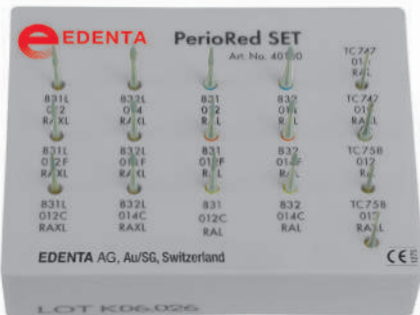
PerioRed SET 40100SO

Calculus-free tooth roots are an important factor for periodontal health.

This SET with 12 diamond- and 4 carbide instruments (TC747/758) helps you with the power-assisted root surface preparation.

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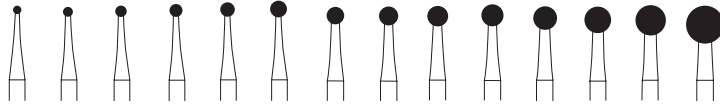
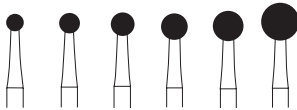
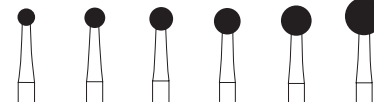
Bone Cutter C141AC

Bone cutter used for gentle preparation of the bone structure. The new optimised A blades guarantee very quiet instrument running with the highest cutting efficiency and minimum trauma of the bone structure to be machined.

The cutter meets the high demands placed on a high-quality surface during surgical preparation of the bone, due to the ZrN (zircon nitride) hard coating. The transitional blade also enables axial insertion in the bone structure. The newly developed blade design guarantees controlled preparation and a smoother bone structure surface for good wound healing.

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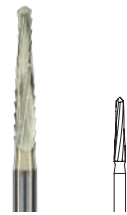
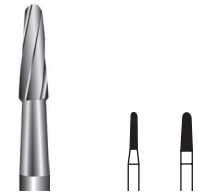
High-efficiency cutting design
for vibration - reduced prepara-
tion.

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ZrN (zircon nitride) hard coating. Indications: For preparing bone and tooth structure. Separating teeth and tooth roots, exposing and removing impacted wisdom teeth, removal of root remnants, bone preparation.

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Indications: Exposing and removing impacted wisdom teeth, apicectomies, preparation of bone structure and a bone lid, separating teeth and tooth roots.

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ZrN (zircon nitride) hard coating. Indications: Exposing and removing impacted wisdom teeth, apicectomies, preparation of bone structure and bone lids, separating teeth and tooth roots.

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Bone cutter for effective, thin bone cuts with only 1.2 mm cutting thickness. Indications: Exposing and removing retained wisdom teeth, apicectomies, preparation of bone structure and bone lids, cutting teeth and tooth roots.

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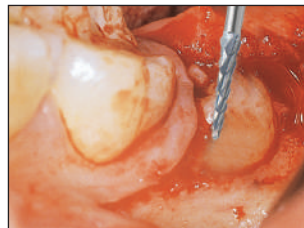
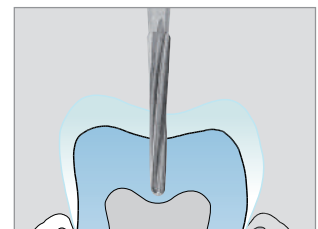
The optimum drill for funnelshaped opening of all pulp chambers.

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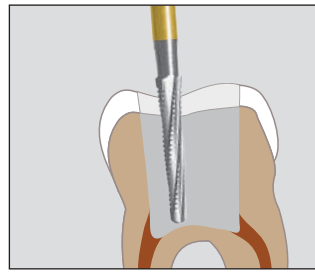
Endo Access Bur C269GKS

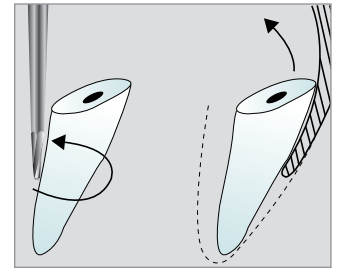
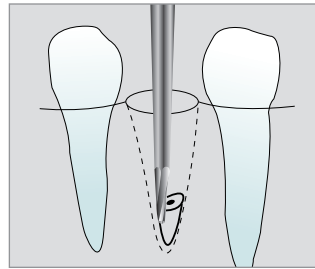
Special endodontic bur with a long, conical blade configuration and cross-cut used for preparing a funnel-shaped access in the pulp chamber. The cross-cut guarantees quiet, efficient removal of tooth structure. The rounded, non-cutting bur tip (safe end) prevents penetration into the root canal walls and pulp chamber floor. Application after removal of the pulp chamber roof using the endodontic bur 802L for extending the access opening to ensure successful endodontic treatment.

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Peri-implantitis treatment

Tungsten carbide finisher with shank length FGXL (overall length 30 mm), for surgical peri-implantitis treatment. Used for removal of micro and macro structures on titanium implants as well as smoothing the surface, thus impeding the formation of new plaque.

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Application

Only operate applying minimal pressure (<2N) and sterile coolant fluid. Optimum speed 40.000 rpm in the red-ring contra-angle handpiece. Move the finisher «anticlockwise» around the implant to reduce the possibility of the instrument running off centre during preparation.

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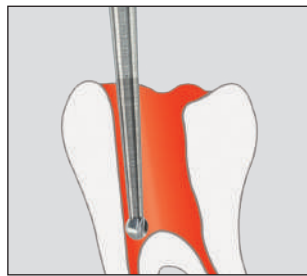
Endobur for root canal preparation

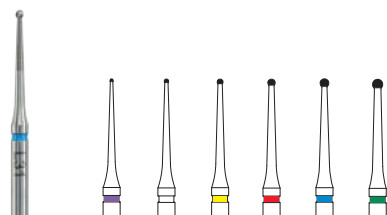
During endodontic treatment of multi-root teeth, the canals cannot always be detected and penetrated immediately. An isthmus must then be prepared over a section or the entire length, so that a hidden canal can be detected. The newly developed Endoburs are ideal for these treatment procedures.

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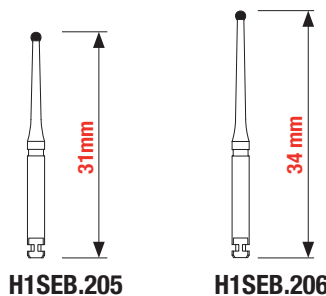
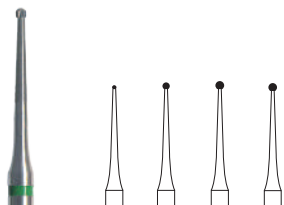
Application:

The efficiently cutting bur blades enable pressureless, minimally invasive preparation. The long, slender neck guarantees good visibility to the access cavity. Endoburs with L34 are particularly suitable for preparing under a microscope to ensure an optimum view of deeper areas of the pulp chamber, and showing the anatomy of the cavity chamber floor. This facilitates minimally invasive opening of root canal orifices and exposure of obliterated canals.

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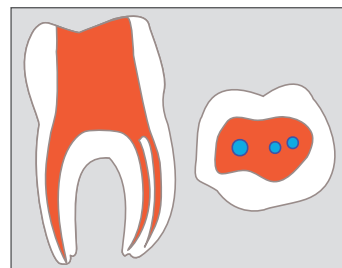
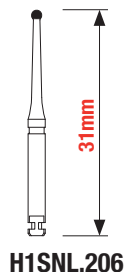
Efficiently cutting special endodontic bur used for detecting all root canals in the case of multirrooted teeth. The H1SNL has a particularly high cutting efficiency and its long neck ensures a clear view in deeper regions of the cavity for opening the access cavity of the root canal system. Use in a contra-angle with micromotor at 1,500 – 2,000 rpm, without cooling, dry and apply only light pressure during preparation.

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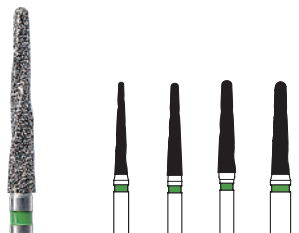
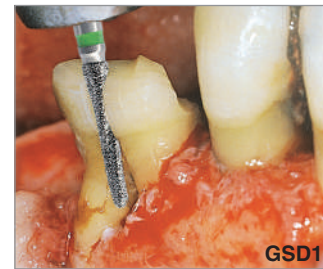
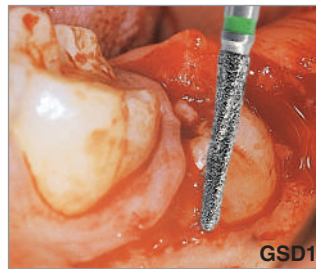
Diamond coated bone cutter for bone incisions, removal and precision finishing works as well as for severing tooth-roots.

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Advantages:

The combination of special shaping and diamond coating results in close contact with bone structure, less heat development as well as a better removal of bone chips and tooth structure.

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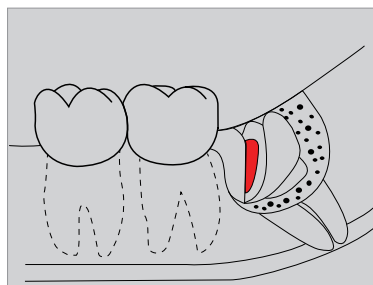
Surgery diamond instrument used for removal and smoothing of bone surfaces as well as separation of teeth and effective machining of the tooth structure in tooth osteotomy.

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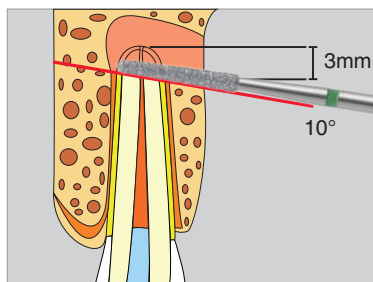
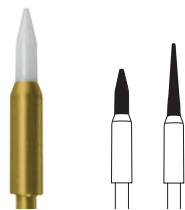
Surgery diamond instrument used for atraumatic separation of teeth and effective machining of the tooth structure during apicectomy. Not recommended for machining bone.

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300.000 - 500.000

Tissue Trimmer STT :

For gingival treatment. As an alternative to using scalpel or electro-surgery.

Designed for soft-tissue surgery, the STT ceramic tissue trimmer is a rotary instrument that can be used as an alternative to electrosurgery and surgical blades. The biocompatible hard oxide point is said to cut tissue without burning.

Applications include modeling gingiva, opening the sulcus for impression, freeing deep cavities, lacerating inter-radicular granulation-tissue, removing gingival hyperplasia/papillectomies, and exposing intra-osseous implant sections and retinated teeth.

Advantages:

The special porcelain of the working section separates the tissue and reduces bleeding (coagulation).

Reduced heat build-up prevents necroses during treatment.

The high stability of the Soft Tissue Trimmer allows rapid, safe gingival preparation.

To be used without any water/spray cooling!

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Soft Tissue Trimmer:

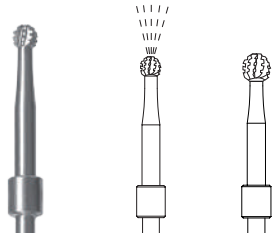
- ① Gingivectomy (eliminating hyperplastic gingiva)
- ② Recovering implants
- ③ Inter-radicular debridement to remove granulated tissue
- ④ Excising mucosa to expose impacted/partially impacted teeth

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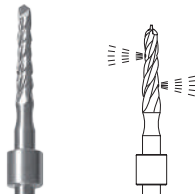
Bone cutter with internal cooling (IK) for gentle preparation of bone structure, stainless steel.

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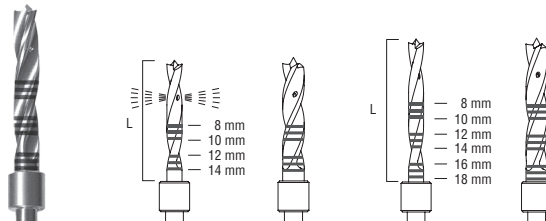
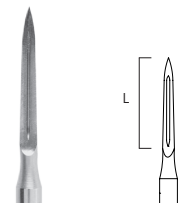
Pilot bur with internal cooling (IK) for implantology, stainless steel with depth marking = 2 mm

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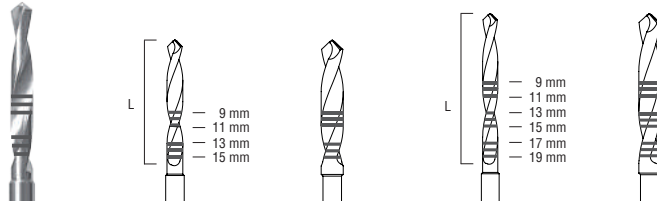
Initial Bur 186RF

The initial bur is used as the first instrument during surgical drilling procedures to place a drill mark and thus prevent the subsequent spiral drill skidding, in particular with tapering alveolar ridges.

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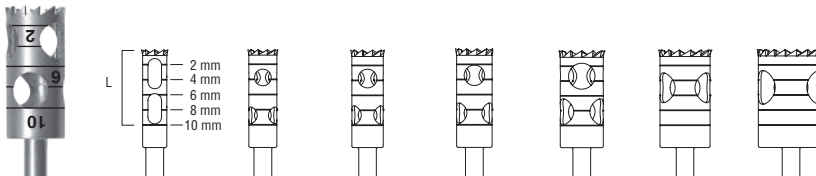
Pilot Bur 210L

The efficiently cutting pilot bur is suitable as a pre-drill for all commonly used implant systems for initial preparation of the implant site. The specially designed bur tip and the good axial drilling properties enable precise drilling. The large chip escape channels guarantee rapid chip removal. The lasered depth marks at intervals of 2 millimetres provide exact visualisation of the penetration depth.

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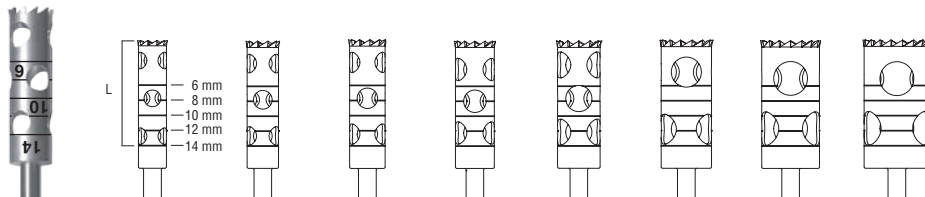
Trepan bur 229, 229L

Specially suitable for use in oral surgery and in implantology for accurate, easy preparation of cylindrical bone grafts and bone extraction (bone lid during apicectomies). The good axial drilling properties enable precise drilling. The efficiently cutting bone blades allow atraumatic treatment. The lasered depth markings 2 millimetres apart provide exact visualisation of the penetration depth.

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Steel instruments for precise preparations

Every preparation is an integrated procedure. With an Edenta preparation set of instruments the procedure flows smoothly from one working stage to the next, with fewer instrument changes and a shorter preparation time.

The steel range provides universal precision instruments suitable for many applications, from cavity preparations to surgical treatment.

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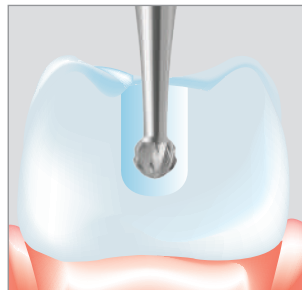
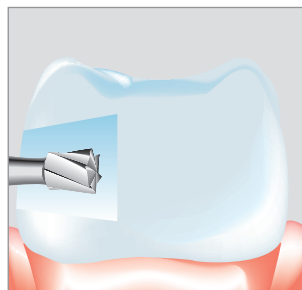
Exkavation, Retention, Perforation, Penetration, Trepanation.
Excavation, retention, perforation, penetration, trepanation.
Excavation, rétention, perforation
Pénétration, trépanation.

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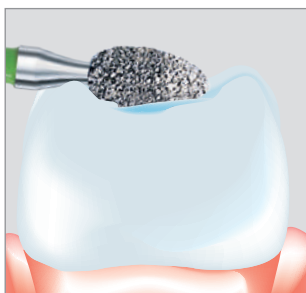
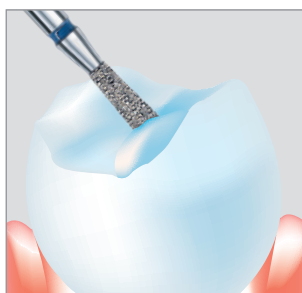
Diamond instruments for precise preparations

Every preparation is an integrated procedure. A coordinated preparation set of instruments aids preparation with fewer instrument changes and a shorter preparation time.

The diamond instrument range provides universal precision instruments for many applications, including cavity preparations, crown abutment preparations, veneer preparations and periodontal treatment.

Okklusale Präparation.
Wiederherstellung der okklusalen Oberfläche.
Occlusal preparation.
Restoration of the occlusal surface.
Préparation occlusale.
Restauration de la surface occlusale.

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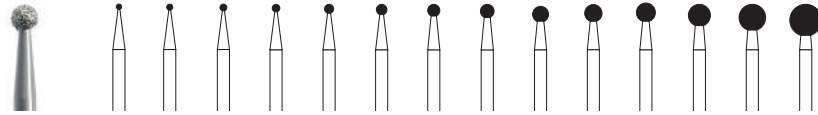
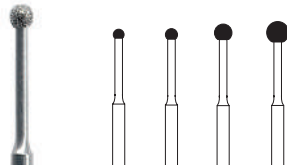
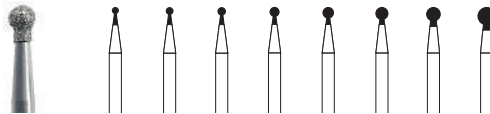
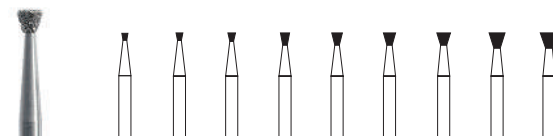
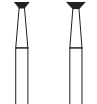
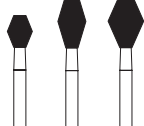
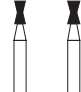
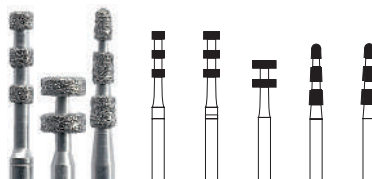
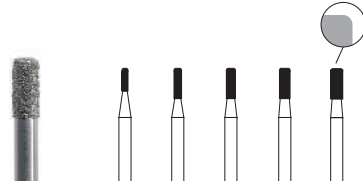
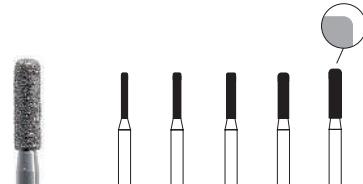
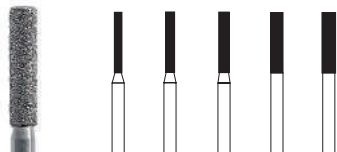
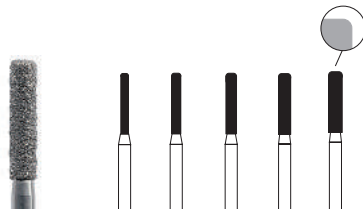
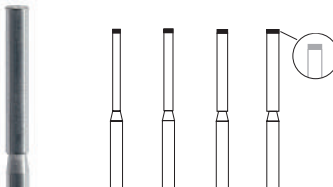
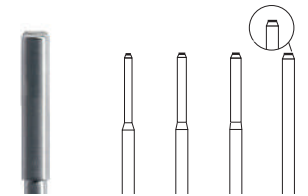
GF844 TwoGrit Diamond

Diamond instrument used for preparing and finishing without changing the instrument when preparing for a veneer. The working section of the TwoGrit diamond instruments consists of two different diamond grits.

The main conical section is coated with coarse diamond grit and the tip with fine diamond grit.

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KS Diamond instruments for preparing a tooth for a crown

The multi-layered diamond coating guarantees optimum material reduction with efficient chip flow ensuring high reduction of tooth structure, thus saving time during treatment as well ensuring reduced stress for the patient.

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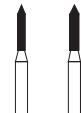
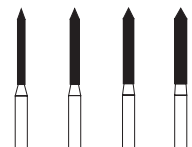
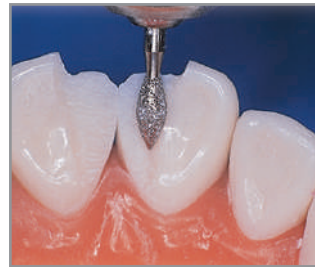
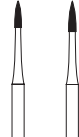
899
Knospenspitze
Pointed Bud
Bouton pointu

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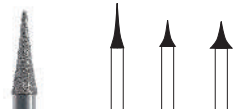
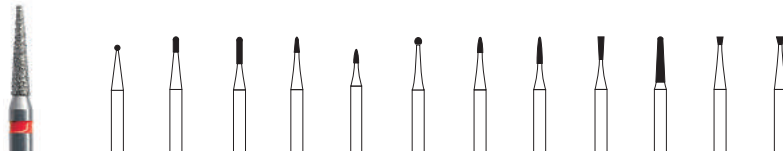
Occlusal shaper 972

The Occlusal shaper 972, which was specially developed for occlusal contouring, is suitable for all filling materials and also all-ceramic ZrO₂ material for reliable contouring of the cusps and fissures using only one instrument. The round shape facilitates contouring of cusps and fissures. The curvature prevents cutting of notches. The fine and superfine grit produce a perfect surface for the subsequent polish.

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The perfect instrument for occlusal contouring (occlusal design) of composite restorations in the posterior region. An instrument for the complete contouring of occlusal surfaces. For quick trimming of a perfect relief on all composite filling materials. Three instrument sizes enable contouring on all premolars and molars.

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Depth Marker TM for gauging the depth

The new depth markers indicate the required preparation depth before preparation. The preparation depth is lasered on the shank. Depth markers for areas of crown and bridge work such as ceramic crowns, overlays and onlays. The depth marker is ideal for preparing shallow depths (0.5 – 1.0 – 1.5 – and 2 mm). It has a high-grade diamond-coated working section with subsequent wide neck, which automatically stops deeper penetration with the depth marker, preventing an excessively deep preparation. This guarantees a high degree of reliability for the operator, also particularly in the case of CAD/CAM restorations which place high demands on preparation.

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The guide pin acts as a spacer and provides for uniform depth of cut as well as controlled, minimal reduction of the tooth structure while preventing excessive preparation.

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All-porcelain restorations

Natural aesthetics and biocompatibility are advantages provided by all-porcelain restorations. They require a broad, uniform preparation margin and maximum conservation of tooth structure. These diamonds with guide pins are available in standard grit for contouring and fine grit for finishing thus providing for optimum shoulder preparations.

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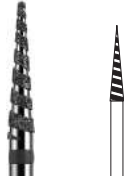
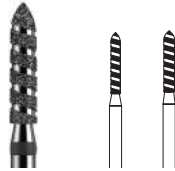
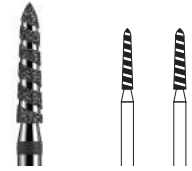
PerioRed Instruments

for gentle root cleaning and root polishing.

Application hints: light working pressure (0,5N), work always with cooling liquid (min. 50ml).

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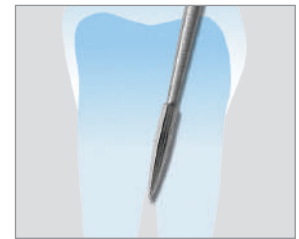
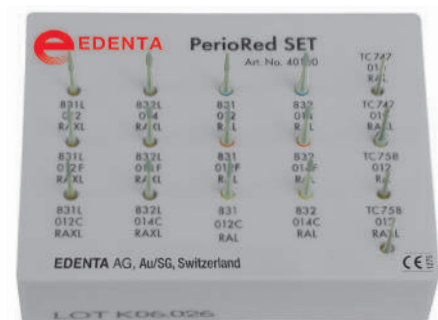
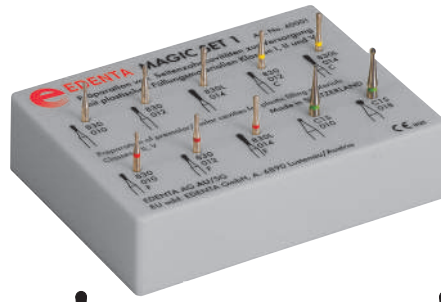
Preparation of premolar and molar cavities for plastic filling materials.

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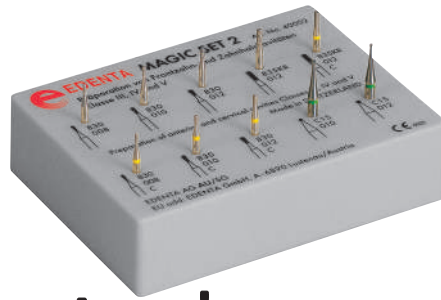
Preparation of anterior and cervical cavities.

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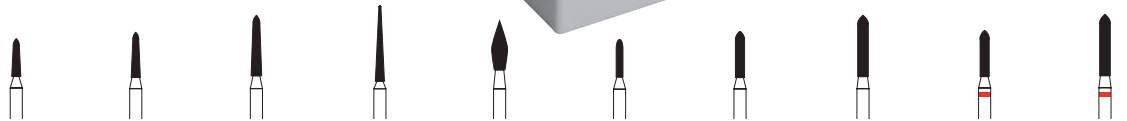
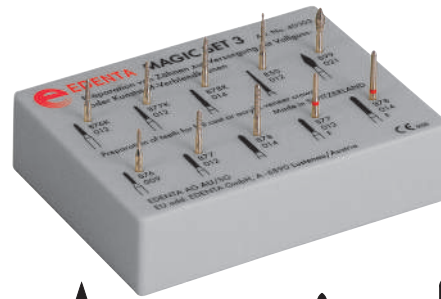
Preparation of teeth for full cast or acrylic veneer crowns.

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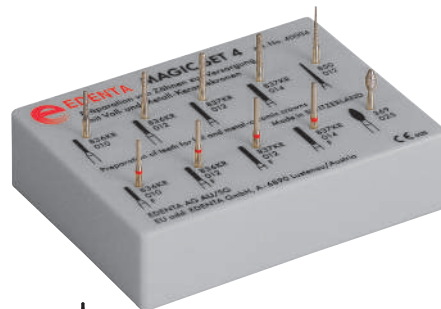
Shank	L mm	5,0	6,0	8,0	10,0	6,5	5,0	6,0	8,0	6,0	8,0
FG	Order No.	876K.314.012	877K314.012	878K.314.014	850.314.012	899.314.021	876.314.009	877.314.012	878.314.014	F877.314.012	F878.314.014
	Seite - Page	84	84	84	80	85	83	83	83	83	83

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Preparation of teeth for full and metal-ceramic crowns.

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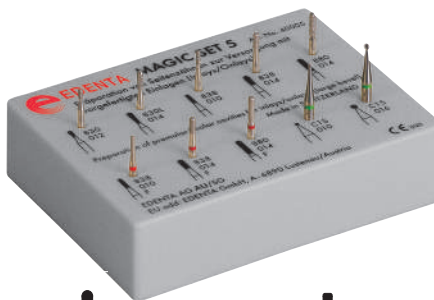
Preparation of premolars and molars for inlays and onlays.

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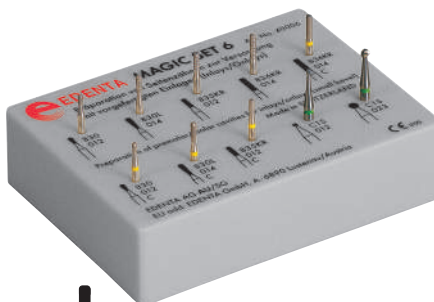
Shank	L mm	2,7	5,0	4,0	4,0	6,0	2,7	4,0	6,0	C1S.314.010	C1S.314.016
FG	Order No.	830.314.012	830L.314.014	838.314.010	835KR.314.012	880.314.014	F830.314.010	F835KR.314.012	F880.314.014	C1S.314.010	C1S.314.016
	Seite - Page	76	76	78	77	84	76	77	84	40	40

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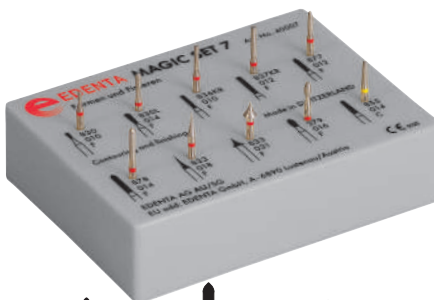
Crown Prep-Set, diamond instruments for the preparation of CAD/CAM fabricated ceramic crowns

CAD/CAM fabricated all-ceramic restorations require a ceramic-specific preparation with rounded edges and without sharp junctions. The appropriate preparation diamonds are contained in the Crown Prep set.

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All-ceramic ZrO2 preparation with diamond instruments using water cooling.

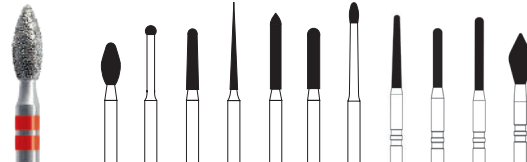
The Multilayer diamond instruments guarantee a very long operating life on hard all-ceramic materials. The newly developed technology of fabrication with a special matrix-bond prevents the breaking out of diamond particles and increases multiple times the operating life of the instruments. The fine grit size provides an optimal cutting performance and high trimming capacity during the separation with minimum vibrations.

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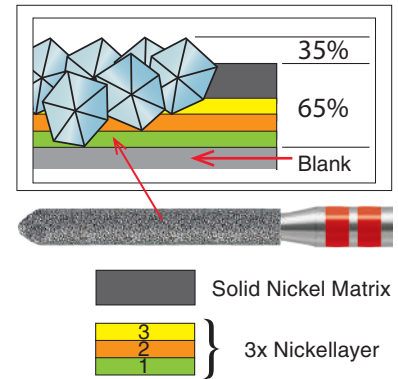
Application:

- Occlusal adjustment
- Adaptation of all-ceramic Abutments
- Trepanation
- Removing an all-ceramic crown
- Fitting all-ceramic restorations

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Trepanation trough allceramic crown

An access cavity is prepared in two stages. ① The facing porcelain is first removed with a diamond instrument without perforating the porcelain framework. ② The porcelain framework is then perforated with a diamond instrument leaving a gap of 0.5 mm from the bonded porcelain (to prevent bonded porcelain breaking off).

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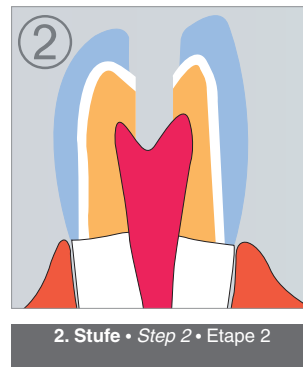
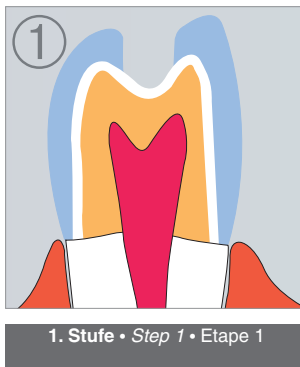
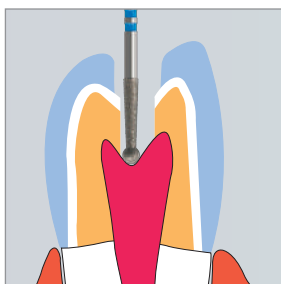
Trepanation with K802L ZirTec- Diamond

K Diamond drill used for trepanation of all-ceramic zirconia restorations. The ZirTec diamond drill makes it easier to produce a straight access to the root canal and to create a funnel shaped access to the pulp chamber.

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K-Diamonds with 80µm special grit

Zircon oxide abutments are the perfect solution for all-ceramic superstructures. The abutments are contoured using the newly developed K-Diamond instruments. The 80 µm special grit prevents damage of the zircon oxide all-ceramic. Crack formation and breaking off of crystals from the zircon oxide is prevented.

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K879

Torpedo
Modified Beveled Cylinder
Torpille

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Multilayer diamond instruments with a special coating.
 Maximum retention of the natural diamonds ensures a very long service life for the instrument. Self-cleaning due to rapid removal of the trimmed material.
 Cool grinding. Smooth ground surface. Atraumatic, rapid preparation.

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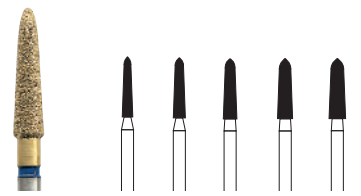
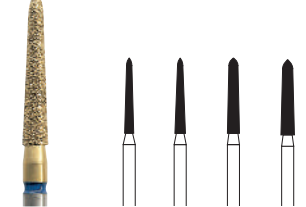
VT879

Torpedo
Beveled Cylinder
Torpille

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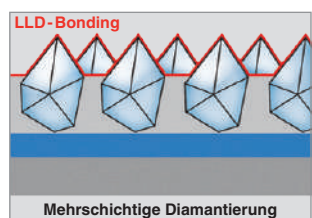
Fig. 1

The diamonds of the WhiteTiger diamond instruments are multi-layered and permanently bonded to the stainless steel shank using a LLD manufacturing process.

The high quality of these diamond instruments offers the user many advantages compared to working with standard diamond instruments and leads to optimal results with all preparations.

High cutting performance (Fig. 1/3)

With the LLD bond and multi-layered diamond veneer, the tooth structure is reduced faster (high cutting performance), thus ensuring minimum heat generation when cutting (atraumatic preparation).

Smooth cut (Fig. 2)

Conveying the debris away from the surface rapidly enables White-TIGER instruments to rotate concentrically and vibration-free (smooth cut).

Fig. 2

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☞ 3 Stück / Packung

Maximum life-span (Fig. 3)

The LLD bond ensures that diamond particles no longer come loose. This increases the life span of the WhiteTIGER diamond instruments several times over. Using standard diamond instruments the diamond particles come loose much sooner from the bond resulting in a short life span of the instruments.

Application

In restorative and cosmetic dentistry for:
Crown preparation
Cavity preparation
Porcelain and acrylic facing work
Fine-finishing composites

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☞ 3 pièces / boîte

Schulter labial / Shoulder from labial

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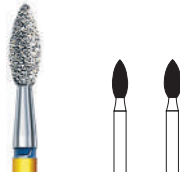
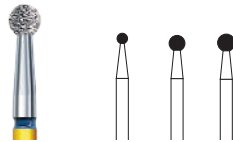
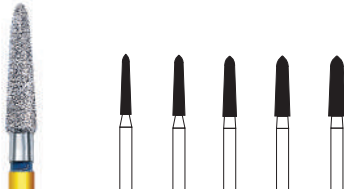
W899

Knospenspitze
Pointed Bud
Bouton pointu

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Applications

- For contouring the proximal regions of all types of fillings
- Removing excess material from the proximal regions of filling-materials like composites, acrylics, glass ionomer cements and amalgam
- Removing excess cement after placing inlays, onlays and laminate veneers

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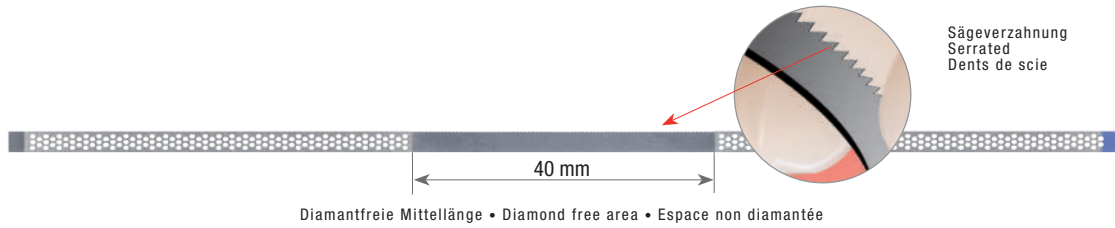
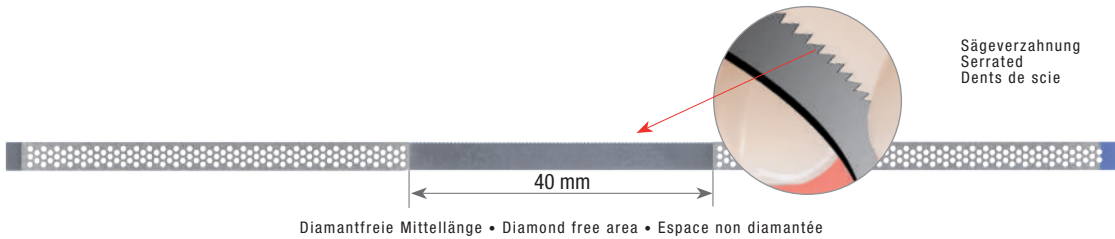
Advantages

- Contours the proximal regions anatomically
- Simplifies working procedures through the serrated edge which separates interdental spaces quickly
- The serrated diamond strips have wider non-diamond coated area, which simplifies sawing
- Serrated sawing edge and diamond-coated abrasive area combined in one instrument simplifies working procedures
- Made of highly stable stainless steel to resist breaking and stretching, for an economical service life
- Autoclavable

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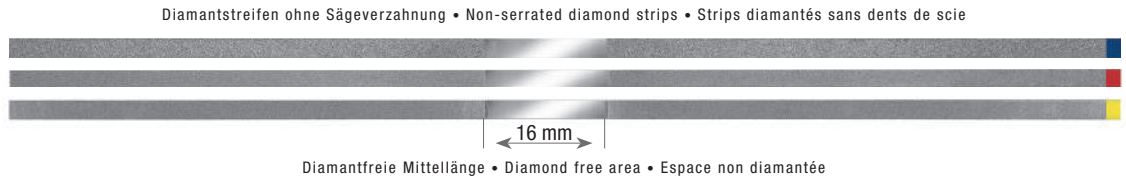
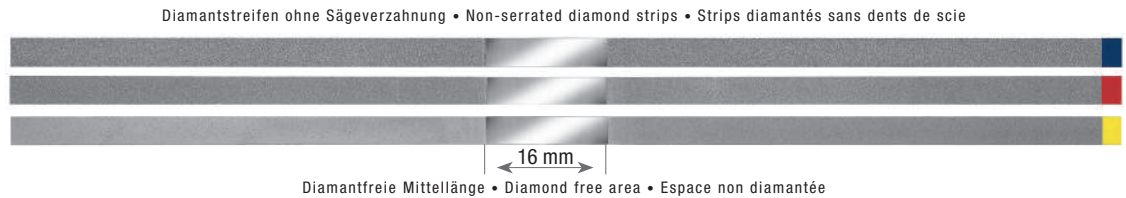
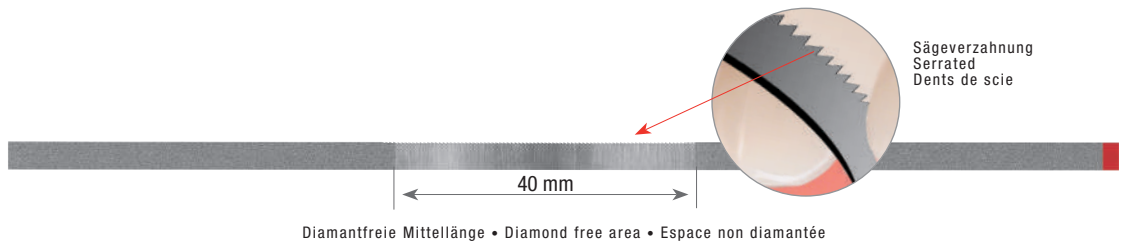
Gain some space by gently contouring the sides of the overlapping teeth. Following interproximal reduction, teeth have sufficient room to align.

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**StripFIX finishing strip system** (handle for diamond strip holder with click-plug system)

Different diamond strip holders are available, which are simply clamped between the fingers and make working in the mouth extremely difficult because of the space availability. The newly developed StripFIX finishing strip system makes working with diamond strips easier and more controlled. A user-friendly, ergonomic handle with special grip recess ensures enhanced control for reliable use of the StripFIX finishing system. The click-plug system enables easy, quick replacement of the diamond strip holders. The swivel-mounted diamond strip holders can each be locked in 4 positions. This ensures that the optimum working position (angle) of the diamond strip is fixed to enable easy access to the anterior and posterior teeth in the oral cavity. All components of the StripFIX finishing system are autoclavable and re-usable.

Application

Used for removing cement after cementation of crowns, bridges, inlays, onlays or veneers. Used for finishing and polishing interproximal fillings and for removing excess filling material. For dental hygienists to enable removal of discoloration that cannot be accessed with their instruments. In interproximal reduction (IPR), used for reducing interproximal enamel during orthodontic procedures.

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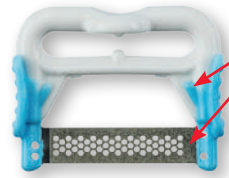
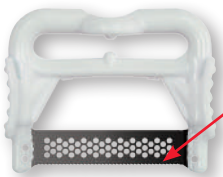
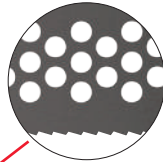
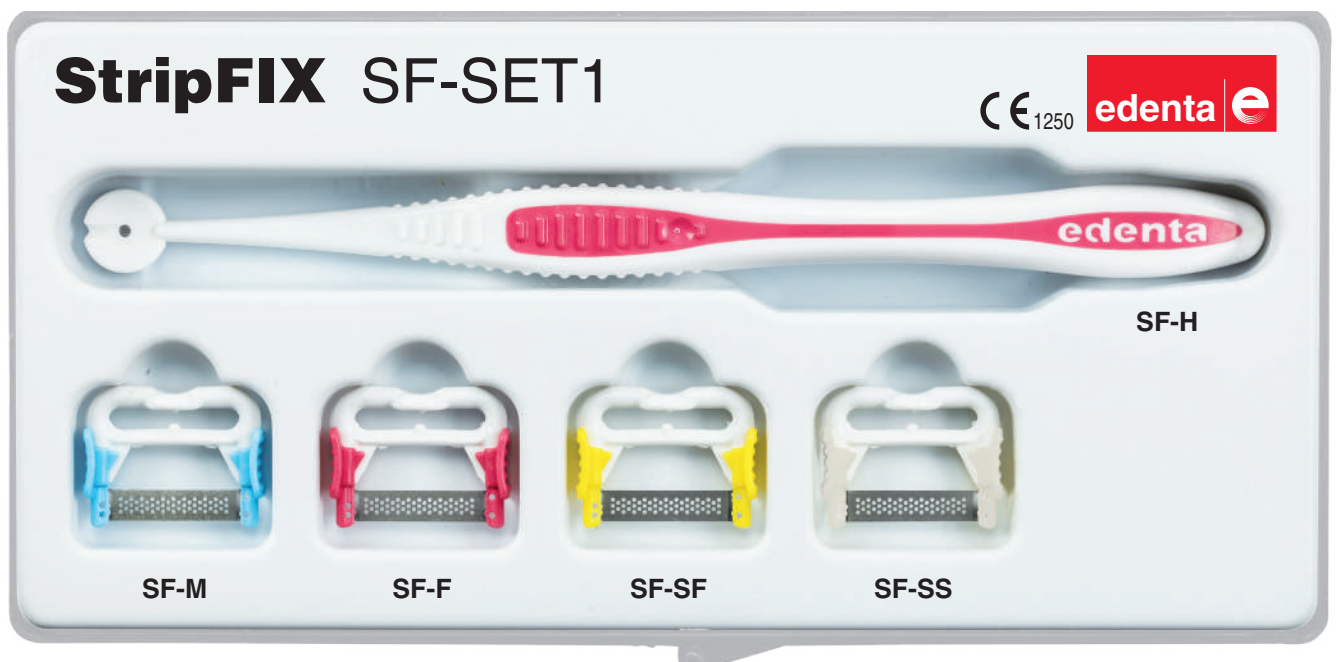
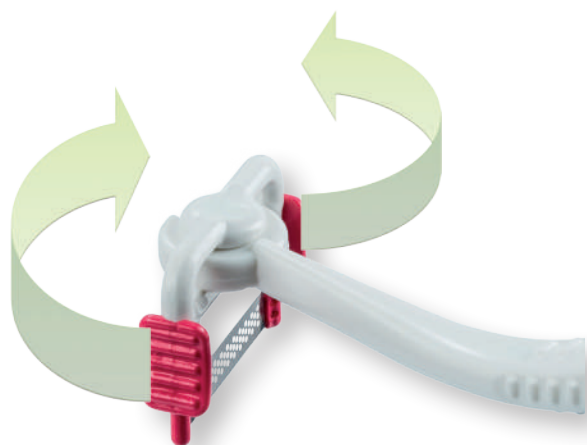
Art.-No. SF-H

StripFIX handle (Art. No. SF-H) for diamond strip holders with click-plug system

The user-friendly, ergonomic handle with special grip recess ensures enhanced control for reliable use of the StripFIX finishing system. The click-plug system enables easy, quick replacement of the diamond strip holders.

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StripFIX diamond strip holder positioning

The swivel-mounted diamond strip holders can each be locked in 4 positions. This ensures that the optimum working position (angle) of the diamond strip is fixed to enable easy access to the anterior and posterior teeth in the oral cavity.

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StripFIX diamond strip holders, Application without handle

Diamond strip holders can be used, with or without handle, by simply holding between the fingers. When using without the handle, the index finger should be placed on the diamond strip plate, ensuring greater control for reliable use of the diamond strip.

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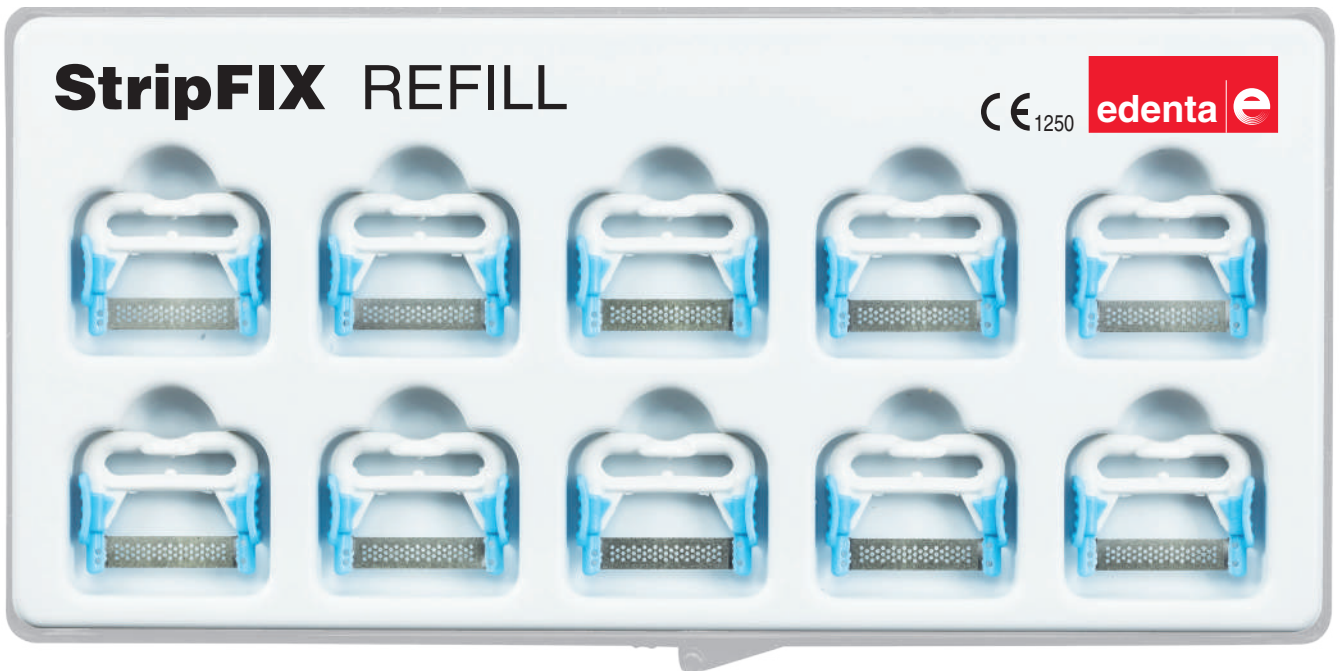
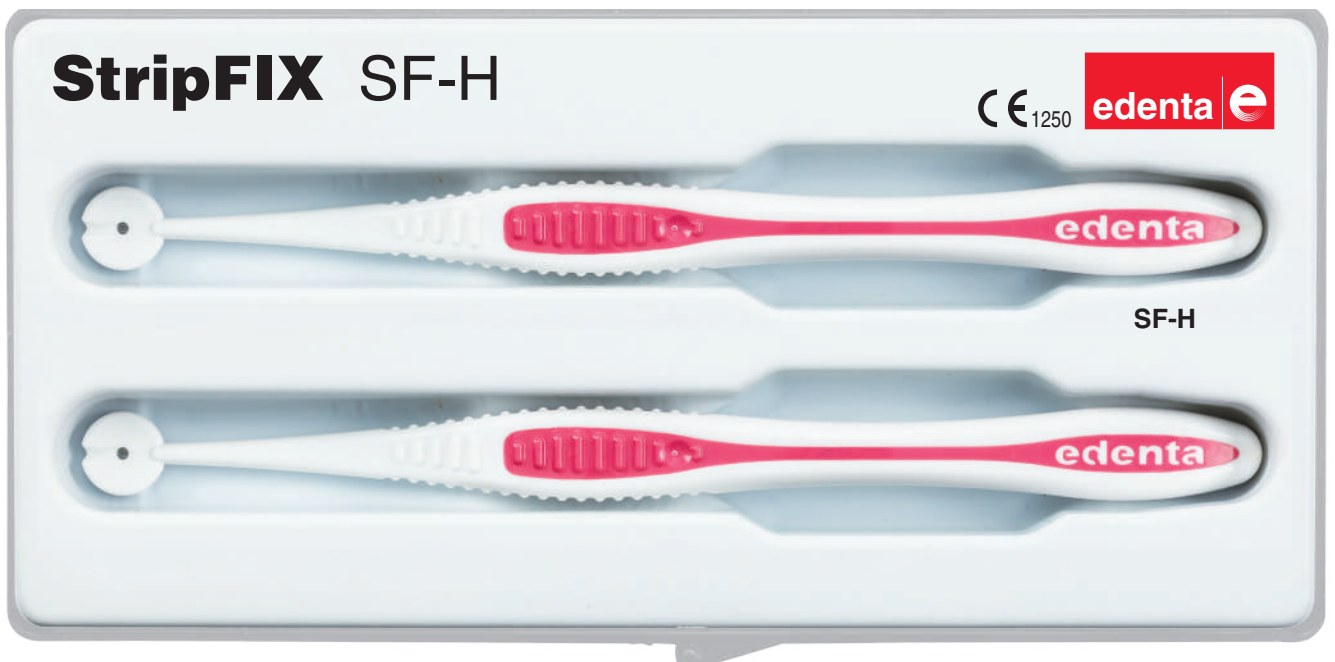
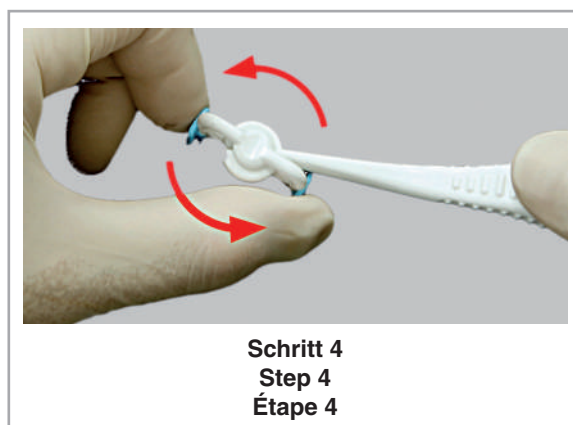
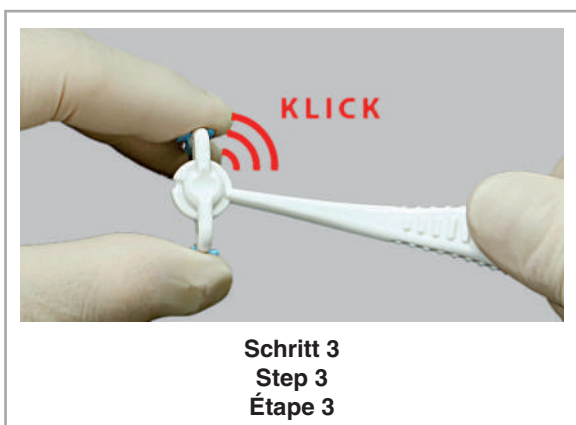
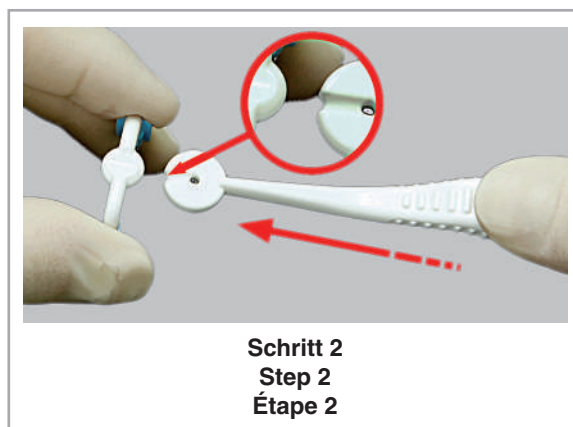
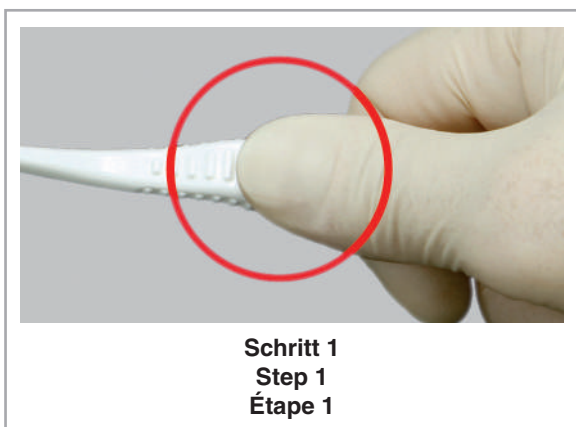
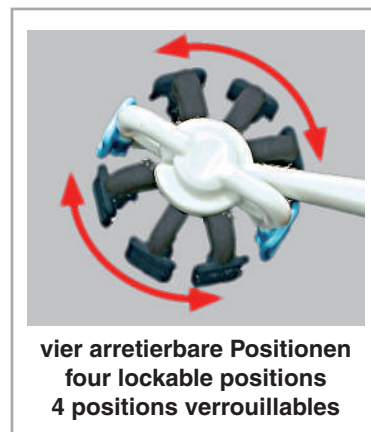
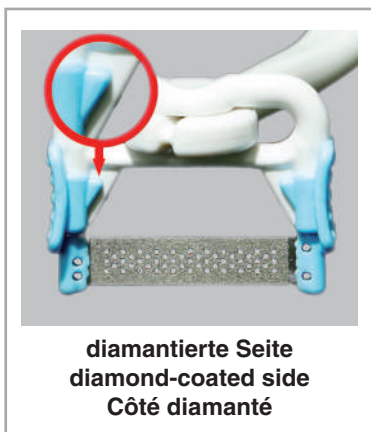
Anwendungshinweise
 Insrtuction for use
 Notes d´application

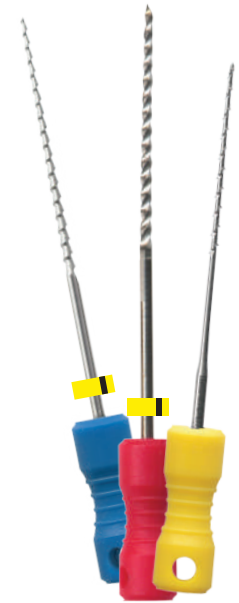
Root canal instruments

Preparation instruments with high flexibility and cutting capacity for endodontics.

All instruments have non-cutting tips (atraumatic tips).

High flexibility provides maximum loading capacity and safety.

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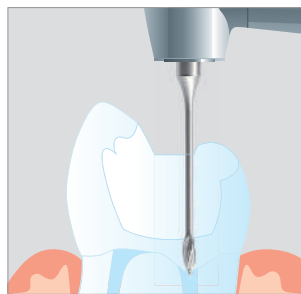
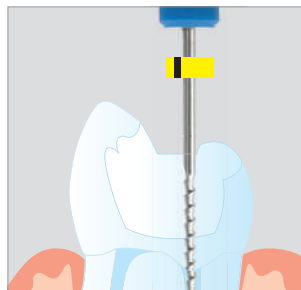
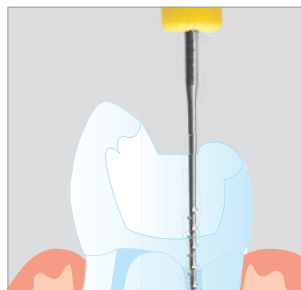
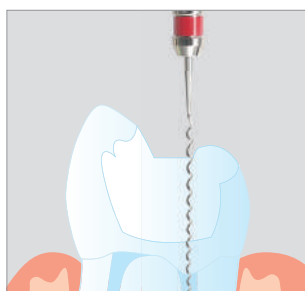
Exstirpation der Pulpa.
Extirpation of the pulp.
Extirpation de la pulpe.

Wurzelkanalfüllung.
Root canal filling.
Obturation canalaire.

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INSTRUMENTS FOR ROOT CANAL PREPARATION
INSTRUMENTS CANALAIRES

	Seite Page
 Gates Gates Gates	114
 Peeso Peeso Peeso	114
 Endobur	114
 Endo-Diamant Endo-Diamond Endo-Diamant	115
 Konus + Sicherheitsspitze Round + Safe End Taper Cône + à extrémité mousse	115
 Endo	115
 Endo	116
 Endo	116
 Nervnadeln Nerve Broaches Tire - nerfs	117
 K-Bohrer K-Reamer Broche „K“	117
 Wurzelkanalfeilen Typ „K“ „K“ - Files Lime „K“	117
 Hedström Feilen Hedstroem Files Lime Hedstroem	118
 NiTi Feilen NiTi Files Lime NiTi	119

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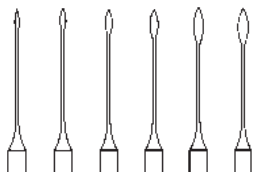
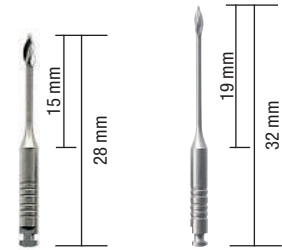
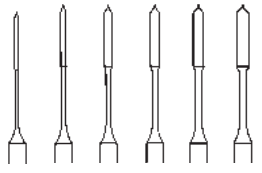
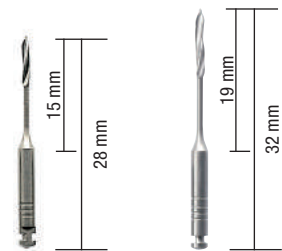
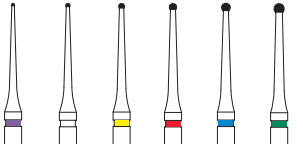
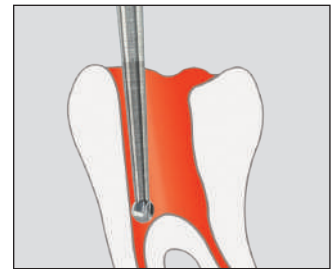
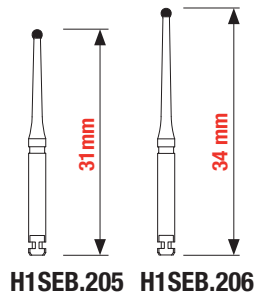
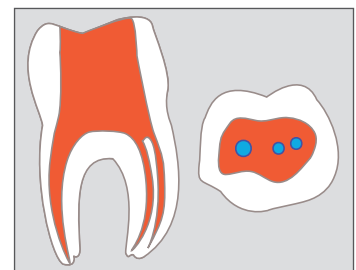
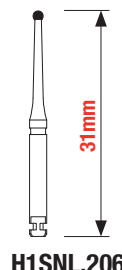
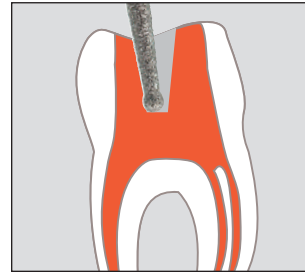
Special endodontic drill for reducing the pulp chamber roof and extending the access opening for successful root canal treatment. The long diamond-coated instrument neck facilitates producing a straight access to the root canal and creating a funnel-shaped access to the pulp chamber. The special diamond grit guarantees rapid and gentle removal of the tooth structure. Speed opt. 120,000 – 160,000 rpm. Apply only light pressure during preparation.

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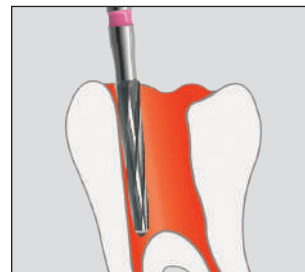
Special endodontic drill with a long, conical blade configuration for producing a funnel-shaped access into the pulp chamber. The rounded, safe-end drill tip prevents penetration into the root canal walls and the floor of the pulp chamber during drilling. Maximum motor speed 120,000 rpm, apply only light pressure during preparation.

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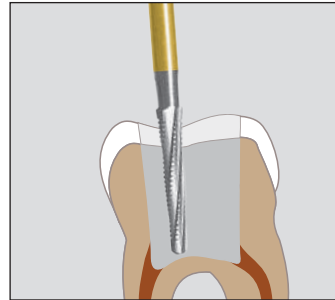
Endodontic drill for reducing the pulp chamber roof and extending the access opening for successful root canal treatment. The long diamond coated instrument neck facilitates producing a straight access to the root canal and creating a funnel-shaped access to the pulp chamber.

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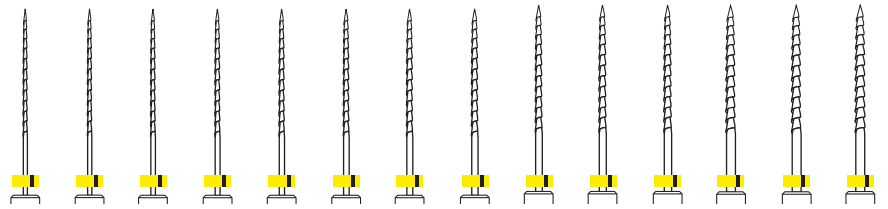
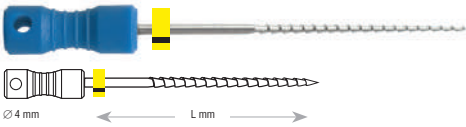
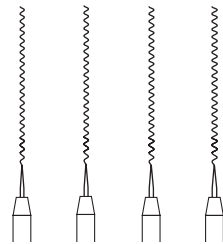
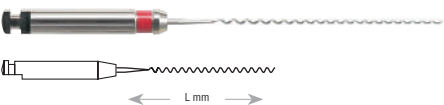
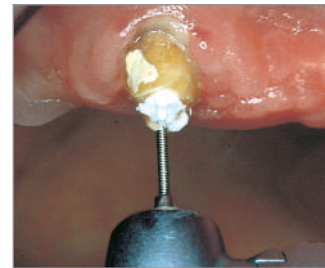
Canal sizes

Three different sets of instruments are available according to the size of the canal; narrow, medium or wide.

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Recommended use

Max. contra-angle speed = 300 rpm. Use a torque-controlled motor:
Torque for .04 = 1.5-1.8 Ncm
Torque for .06 = 2.4-2.9 Ncm
Apply only light pressure during preparation

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Preparation sequence (crown down technique)

- Prepare straight coronal canal access region using file No. 10.
- Create the guide path using a K manual file No. 010/015 and determine the working length (WL).
- Then prepare using file No. 06 and file No. 04 to approx. 3 mm before the working length (WL).
- Apical finishing then follows using file No. 06 and file No. 04.

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Root posts with retentive head.

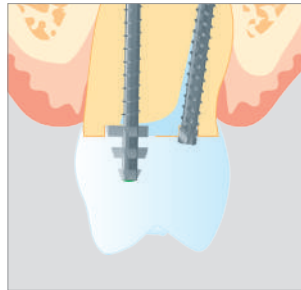
All root posts are made from a titanium alloy with high mechanical strength, are completely corrosion resistant and bio-compatible.

The design of the root posts ensures uniform force distribution, allowing stress-free fitting.

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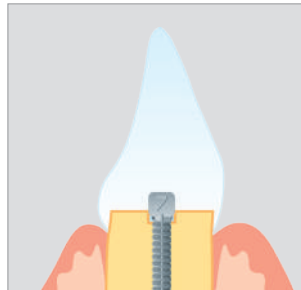
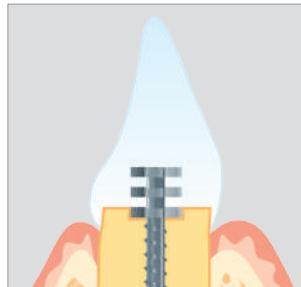
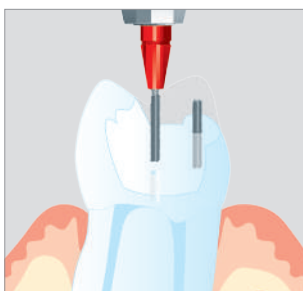
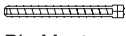
ROOT POSTS
SYSTEMES DE TENONS

	Seite Page
 Head Master, aktiv, konisch Head Master, active, tapered Head Master, active, conique	122
 Head Master, passiv, konisch Head Master, passive, tapered Head Master, passive, conique	123
 Head Master, aktiv, zylindrisch Head Master, active, cylindrical Head Master, active, cylindrique	124
 Head Master, passiv, zylindrisch Head Master, passive, cylindrical Head Master, passive, cylindrique	125
 Pin Master, aktiv Pin Master, active Pin Master, active	126
 Post Master, passiv Post Master, passive Post Master, passive	127

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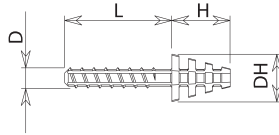
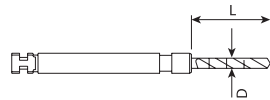
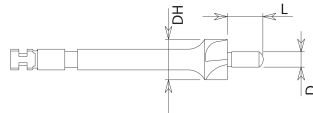
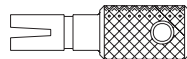
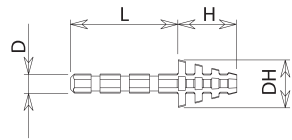
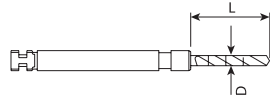
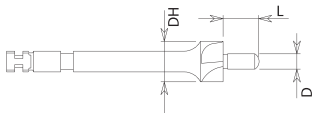
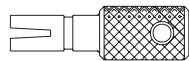
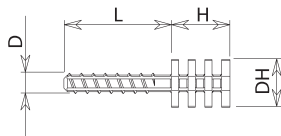
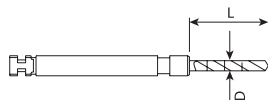
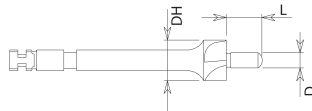
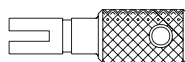
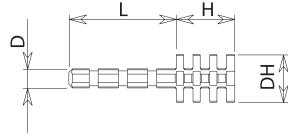
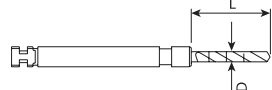
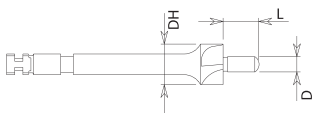
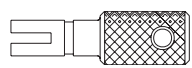
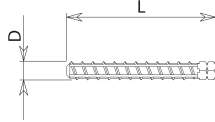
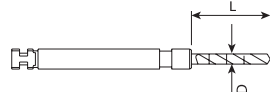
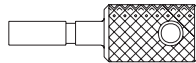
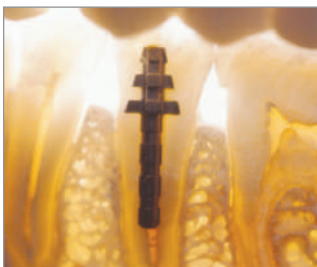
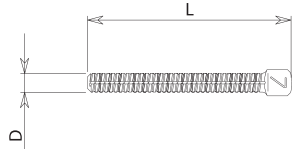
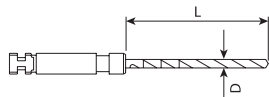
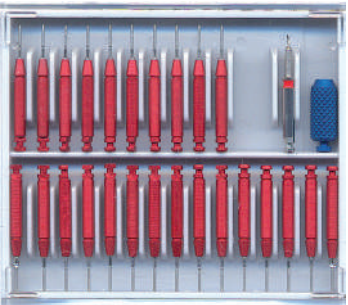
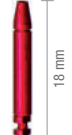
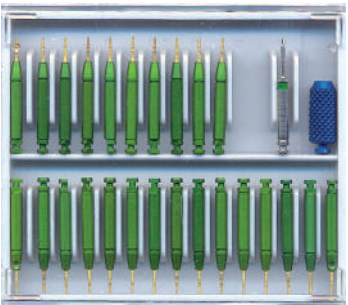
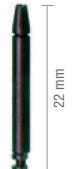
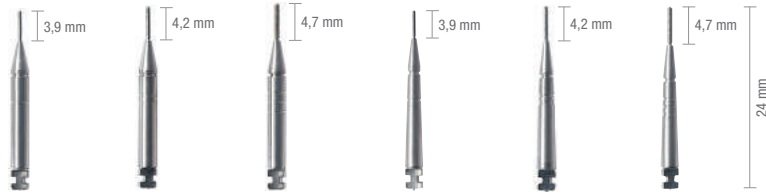
.017" / 0.5 mm	.021" / 0.6 mm	.027" / 0.8 mm	.017" / 0.5 mm	.021" / 0.6 mm	.027" / 0.8 mm
Order No.	Order No.	Order No.	Order No.	Order No.	Order No.
2013 S0	2016 S0	2018 S0	2021 S0	2023 S0	2025 S0
TriStar			TriJet		
2012 S0	2015 S0	2017 S0	2020 S0	2022 S0	2024 S0

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Handle blue permits manual insertion of the pins

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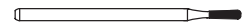
Any FG - Instrument can be converted to a handpiece (2,35 mm ∅) by means of this FG - Adapter.

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FG Adapter with Spring, RA Shank (2,35 mm ∅), stainless steel.

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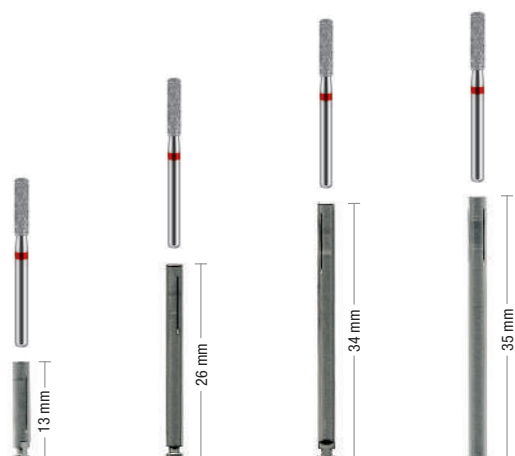
Fast and safe insertion of your FG instrument into the RA or HP adapter, by simply pushing it in. The adapter shaft is constructed like a snap-on clamping sleeve. Once inserted, the instrument will remain in the adapter until it has become dull. Made from special stainless steel, the adapter together with the instrument can be sterilized in any normally used solution or in the autoclave.

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Aluminium bur block for use as an instrument organizer in the dental practice and laboratory. The new bur blocks are used to compile sets of instruments for treatment procedures and preparation techniques.

This enables a more ergonomic and hygienic treatment procedure. A swivel lid prevents the instruments from falling out. All bur blocks can be cleaned, disinfected and sterilized in an autoclave.

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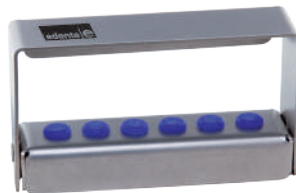
40600

Abmessungen / Dimensions	72 x 20 x 50mm
For 6 FG, HP or RA Instruments / max. length: 47mm	

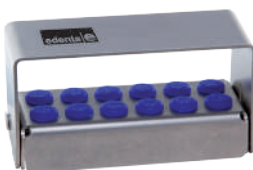
40601

Abmessungen / Dimensions	72 x 20 x 40mm
For 6 FG or RA Instruments / max. length: 37mm	

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40602

Abmessungen / Dimensions	61 x 25 x 30mm
For 12 FG or RA Instruments / max. length: 28mm	

40603

Abmessungen / Dimensions	42 x 25 x 30mm
For 8 FG or RA Instruments / max. length: 28mm	

Bur blocks

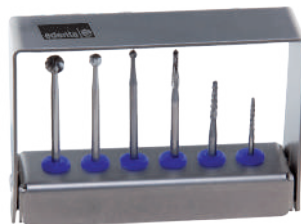
The new bur blocks are manufactured entirely from stainless steel, making them suitable for all types of preparation. The instruments can be cleaned and disinfected in the block either in an instrument solution or also in an ultrasonic cleaner.

They are then sterilised in an autoclave. The instrument shanks are retained in position by universal silicone plugs and so cannot fall out. FG or RA contra-angle instruments can be inserted in the silicone plugs.

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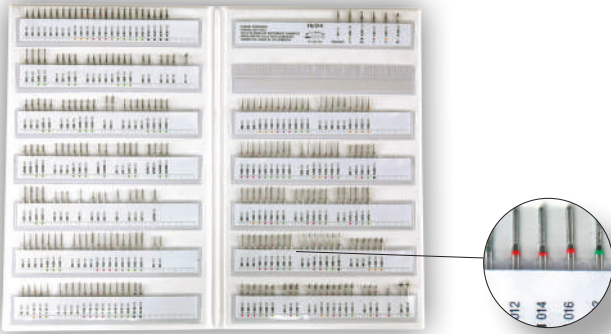
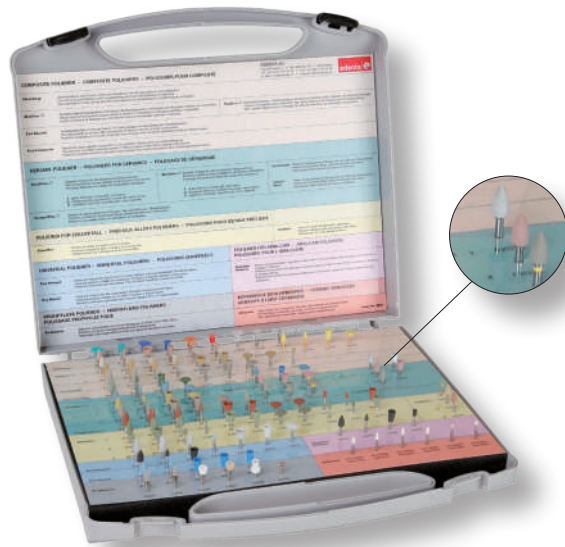
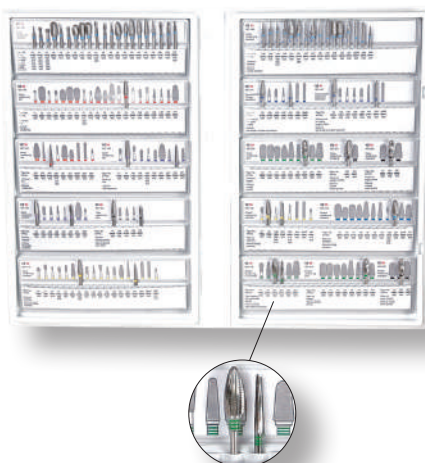
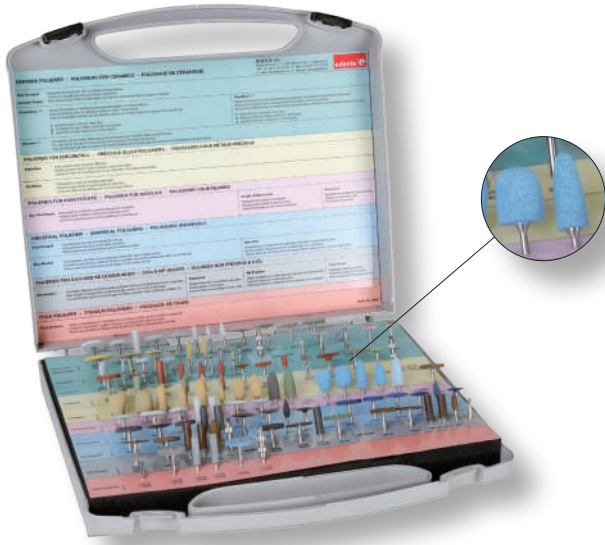
DIAMOND DRESSING STONE

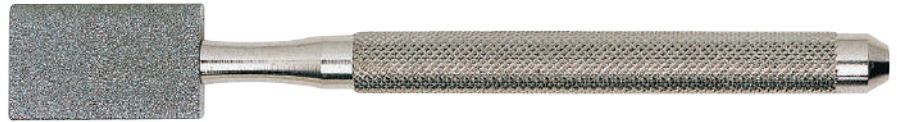
For dressing, shaping and restoring of deformed abrasives, both sides coated.

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CLEANING STONE

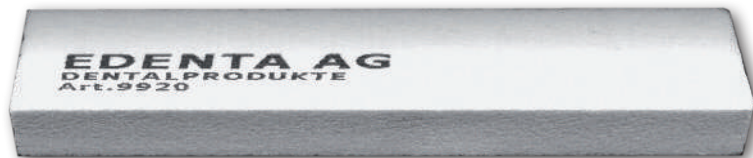
Some maintenance is needed though. Please use our Cleaning Stone No. 9920, for the cleaning of your Sintered Diamond Instruments. It need not to be cleaned too often, but from time to time, this is very important to maintain clean and very sharp cutting edges.

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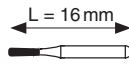
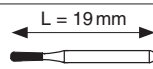
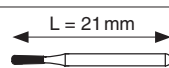
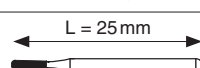
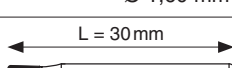
Application:

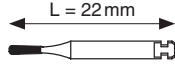
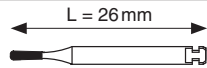
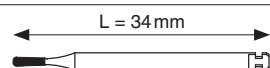
The cleaning stone has to be wet when used. Place the stone in water until no more bubbles rise. The humidity of the block prevents the development of dust and improves the cleaning effect decisively.

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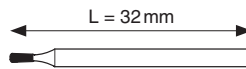
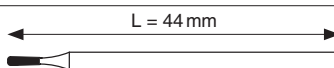
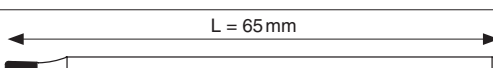
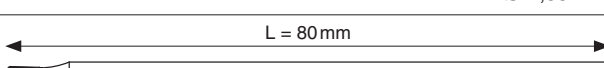
Ordering options

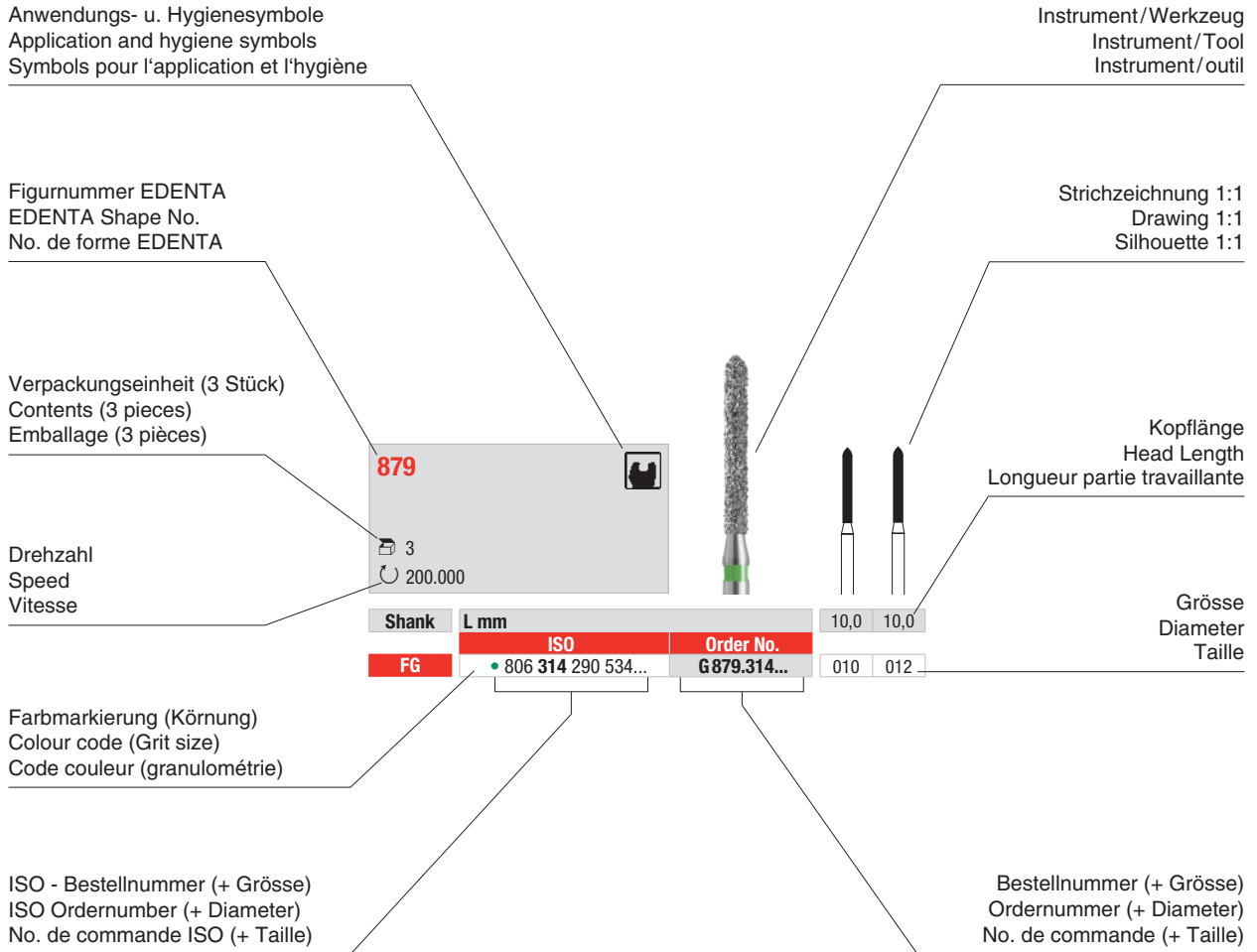
You are free to use the EDENTA order number or the ISO numbering system when placing an order.

Smooth handling of your order is guaranteed in either way.

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Numbering System ISO 6360

The ISO order number consists of a certain number code indicating specific instrument related data for clear identification.

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Nominal size ISO 2157

- Largest diameter of the working part (1/10 mm)

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All instruments have been developed and constructed for their specific use. Improper use can lead to damage to the tissue, premature wear, destruction of the instruments, danger to the user, patient or third parties.

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User Information for rotating dental and surgical Instruments

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- Use of improper instruments can cause insufficient preparation results.
- Excessive speeds while using instruments with overly large heads endanger the stability of the instruments.
- Excessive speeds applied near to the dental pulp can endanger its vitality. The same can be the result when using an inadequate supply of water spray.
- Observe permissible speeds indicated on packaging.
- The handpiece must be in proper working condition. The use of an adequate supply of water spray of a minimum of 50 ml/min on the working part is necessary in the 1500 min⁻¹ speed range to ensure reliable cooling of the material being worked on.
- Additional external cooling is needed for Instruments with a total length of > 19 mm and a head diameter of 1.8 mm (ISO -018).
- Moistening of the surfaces to be prepared must not be impaired by an incorrect suction system. Deflection of the coolant jet can also cause heat related damage.
- The recommended applied contact pressure range is 0.3 – 2 N (30 – 200 p) depending on the kind of preparation.
- Avoid tilting, levering, or blocking the instruments up by using too much contact pressure (increased danger of fracture).
- Insert the instruments as deeply as possible without excessive use of force into the chuck and check to ensure that they are secure.
- Extremely tapered instruments will wear faster at the tip due to the smaller size. To maintain the longevity of these instruments, careful preparation with less contact pressure is important.

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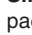
Storage and Keeping Conditions:

- In dry conditions and protected against contaminants. Protect instruments in general against chemicals, acids, heat and extreme temperature variations. Do not use polishers after expiration date indicated.
- **Observe guidelines from Robert-Koch-Institut - ch. E3, (Hygiene Requirements in Dentistry) or the valid legal and hygiene regulations in the respective country!**
- **Steel instruments** are unsuitable for both mechanical preparation and the steam steriliser. They should be replaced by corresponding tungsten-carbide instruments.
- **Single-use products** (identified on the packaging with ) are not approved for reuse. Safe use cannot be guaranteed if these products are reused, as there is the risk of infection and/or the products are no longer safe to use.
- The end of the product's service life is generally determined by wear and damage due to use.
- **Colour-anodised aluminium units** (e.g. Bur block 40500 to 40580 and Retopin mandrel) lose their colour during use of standard cleaning procedures and in a cleaning and disinfection machine. Cleaning and disinfectant agents, which are specifically designed for these materials should be used during preparation.

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Recommendations for use

Polishers / Brushes

- Apply low contact pressure in order to minimize heat generation.
- Polish with circling movements.
- In order to achieve a high-shine polish, for multiple step polishing procedures all polishers are to be used in the indicated sequence.
- Eye protection is recommended.

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Endodontic instruments

- Use instruments ascending in size.
- Don't omit a size.
- As small instruments are more sensitive to strain they should be used only once.
- Large sizes are not suited for curved canals = risk of breakage and perforation.
- Please check the instruments after each use for wear and deformation.
- Protect patients against aspiration. Use rubberdam, secure instruments with safety chain or cord.
- Frequent removal of dentine chips eliminates clogging of the blades. Rinse instruments in 2.5% NaOCl-solution after each use.
- Instruments which are used for the preparation of extremely curved canals should be used only once.

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Nerve broaches 9107 (extirpation needle)

- Push into canal and pull out with a rotary motion of 180°.

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K-burs 171

- Reaming motion in clockwise direction, max. 90°.

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K-files 173

- Filing motion in clockwise direction, max. 45°. Usual methods, step back, step down, balanced force technique.

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Hedstroem-files 174

- Pushing and pulling motion without rotation. Rotational motion would increase the risk of jamming of the sharp blades. In case the canal has been enlarged with K-burs or K-files, the Hedstroem file used subsequently should be of smaller or same size.

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Engine instruments

Gates

- Green contra-angle, 450 - 800 min-1. For preparation of the coronal portion of the root canal, before or after the use of files or K burs.

Peeso

- Green contra-angle, 800 - max. 1200 min-1. Only to be used for opening the canal entrance and for reaming the coronal portion.

Root Canal reamer

- Contra angle, green 800 - max. 1200 min-1. For preparation of the root canal entrance and enlarging the coronal portion.

Instruments for root filling

Root fillers Typ „L“ 178 / 178S

- Green contra-angle. Carefully insert root filler into filling material while the contra-angle is switched off. Stop just before reaching the apex, then start the contra-angle and rotate the filling material in at a speed of max. 800 min-1 and slowly pull out the instrument.

NT Endo-Files

- Adhere strictly to the maximum permissible speed of 300 r.p.m. as exceeding this speed may cause fracturing.
- Use an endomotor-handpiece with controllable torque.
- Maximum torque for .04 taper version: 1.5 - 1.8 Ncm. Maximum torque for .06 taper version: 2.4 - 2.9 Ncm.

Preparation sequence (crown-down technique)

- Prepare straight coronal canal access region using file No. 10.
- Create the guide path using a No. 010/015 K manual file and determine the working length (WL).
- Then prepare to approx. 3 mm short of the working length (WL) using a No. 06 and No. 04 file.
- Apical finishing then follows using a No. 06 and No. 04 file.

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**Manufacturer information (B31)
Medical devices semi-critical B**

For re-processing resterilisable instruments in the dental practice for users with appropriate qualification according to DIN EN ISO 17664.

 www.edenta.com

Downloadcenter => Instructions for use and safety guidelines

**Manufacturer information (B30)
Medical devices critical B**

For re-processing resterilisable instruments in the dental practice for users with appropriate qualification according to DIN EN ISO 17664.

 www.edenta.com

Downloadcenter => Instructions for use and safety guidelines

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Manual Preparation

Manufacturer's information on the reprocessing of resterilisable instruments in accordance with DIN EN ISO 17664 and the recommendations of the RKI Robert Koch Institute).

1 Pre-cleaning

- Place the instruments in an aldehyde-free cleaning/ disinfection solution (e.g. in a bur cleaning stand) immediately after use. Alkaline solutions should **not** be used for the disinfection of polishers and ceramic abrasives. Adhere to the manufacturer's instructions for concentration and reaction time.

2 Cleaning / Disinfection

- Rinse rough contamination under clean running water and remove using a hard plastic brush.
- Clean and disinfect the instruments using a suitable agent (adhere to the manufacturer's instructions for concentration and reaction time), preferably in an ultrasonic cleaner at max. 45°C (risk of protein coagulation). Place the instruments in a suitable instrument holder (instruments should not come into contact).
- As the vibrations in the ultrasonic cleaner can be absorbed by the materials of the polishers and ceramic abrasives, these should only be prepared in an instrument bath using a suitable alcohol-free solution.
- After the reaction time, thoroughly rinse the disinfectant residue from the instruments with clean, running water (use of fully desalinated water in the final rinse stage prevents spots).
- Start the appropriate programme.
- The use of fully desalinated water in the final rinse stage prevents spots.

3 Drying

- Dry instruments and remove residual moisture –if there is any –(preferably with clean, dry compressed air). Concentrate particularly on areas that are difficult to access.

4 Visual check

Check for cleanness and integrity

- if there is visible residual contamination, repeat cleaning and disinfection.
- discard faulty instruments (missing diamond coating, blunt/ chipped blades, fractured work sections, corroded surfaces, bent instruments etc.)

Instruments semi-critical B

Non-invasive use

Instruments come into contact with mucosa or pathologically changed skin.
This manufacturer's information applies for all instruments, which are used for the following non-invasive (preventive/ restorative, dental prosthetic/ orthodontic treatments) measures: these are polishers, ceramic abrasives, rotary ceramic, tungsten and diamond instruments used for cavity and crown preparations, for filling removal and preparation or for crown cutting.

The following sterilisation in a steam steriliser must always be completed with manual preparation of the instruments!!!

Instruments critical B

Invasive use

Instruments, which penetrate the skin or mucosa and consequently come into contact with blood, inner tissues or organs, including wounds.
This manufacturer's information applies for all instruments, which are used for surgical, periodontal or endodontic measures: these are tungsten-carbide and diamond rotary instruments and also rust-proof stainless steel or ceramic instruments as well as steel or nickel-titanium root canal instruments (including manual root canal instruments).

The following sterilisation in a steam steriliser must always be completed with manual preparation of the instruments!!!

5 Steam sterilisation in the fractional vacuum process

(Machine complies with EN 13060, Class B)

- Instruments unpacked in a suitable instrument holder.
- Fractional pre-vacuum (4 times).
- Sterilisation temperature 134°C.
- Hold time 5 minute (full cycle).
- Drying time 10 minutes.

Steam sterilisation in the fractional vacuum process

(Machine complies with EN 13060, Class B)

- Instruments packed in suitable packaging for sterilising goods (see DIN 58952/53 and EN 868).
- Fractional pre-vacuum (4 times).
- Sterilisation temperature 134°C.
- Hold time 5 minute (full cycle).
- Drying time 10 minutes.

6 Storage

- Store protected from dust, moisture and recontamination and observe the storage periods (see DIN 58953).

! Note !

New instruments supplied non-sterile must be prepared before initial use.

Steel instruments are unsuitable for both mechanical preparation and the steam steriliser. They should be replaced by corresponding tungsten-carbide instruments.

Single-use products (identified on the packaging with ) are not approved for reuse.

FlexiSnap mandrels are reusable, can be used in a CDM and are steam sterilisable. **FlexiSnap discs** are single-use products and can **only be manually** (unsuitable for CDM and steam steriliser) **disinfected** using a suitable disinfectant (e.g. HELVEMED Instrument Forte) before initial use.

Colour-anodised aluminium units (e.g. Bur block 40500 to 40580 and Retopin mandrel) lose their colour during use of standard cleaning procedures and in a CDM. Cleaning and disinfectant agents, which are specifically designed for these materials (e.g. HELVEMED Instrument Thermo EC), should be used during preparation.

Mechanical Preparation

Manufacturer's information on the reprocessing of resterilisable instruments in accordance with DIN EN ISO 17664 and the recommendations of the RKI Robert Koch Institute).

1 Pre-cleaning

- Place the instruments in an aldehyde-free cleaning/ disinfection solution (e.g. in a bur cleaning stand) immediately after use. Alkaline solutions should **not** be used for the disinfection of polishers and ceramic abrasives. Adhere to the manufacturer's instructions for concentration and reaction time.

2 Cleaning / Disinfection

- Rinse the instruments under clean, running water before mechanical preparation to ensure that no residue of the cleaning/ disinfection solution gets into the machine.
- Instruments should not come into contact during cleaning, therefore place in a suitable instrument holder.
- Place the instrument holder (the instrument holder should be placed in the cleaning and disinfection machine with the lid open) in the cleaning and disinfection machine (CDM), so that the spray jet is targeted directly onto the instruments.
- Use suitable agent and dosage according to the information on the label and the CDM manufacturer's instructions.
- Start the appropriate programme.
- The use of fully desalinated water is recommended in the final rinse stage to prevent spots.

3 Drying

- When the programme is finished, take the instruments out of the CDM and remove residual moisture –if there is any –(preferably with clean, dry compressed air). Concentrate particularly on areas that are difficult to access.

4 Visual check

Check for cleanness and integrity

- if there is visible residual contamination, repeat cleaning and disinfection.
- discard faulty instruments (missing diamond coating, blunt/ chipped blades, fractured work sections, corroded surfaces, bent instruments etc.)

Instruments semi-critical B

Non-invasive use

Instruments come into contact with mucosa or pathologically changed skin.
This manufacturer's information applies for all instruments, which are used for the following non-invasive (preventive/ restorative, dental prosthetic/ orthodontic treatments) measures: these are polishers, ceramic abrasives, rotary ceramic, tungsten and diamond instruments used for cavity and crown preparations, for filling removal and preparation or for crown cutting.

Subsequent sterilisation in a steam steriliser can be omitted with verifiable disinfection in a cleaning and disinfection machine!!!

Instruments critical B

Invasive use

Instruments, which penetrate the skin or mucosa and consequently come into contact with blood, inner tissues or organs, including wounds.
This manufacturer's information applies for all instruments, which are used for surgical, periodontal or endodontic measures: these are tungsten-carbide and diamond rotary instruments and also rust-proof stainless steel or ceramic instruments as well as steel or nickel-titanium root canal instruments (including manual root canal instruments).

The following sterilisation in a steam steriliser must always be completed with critical B instruments!!!

5 Steam sterilisation in the fractional vacuum process

(Machine complies with EN 13060, Class B)

- Instruments unpacked in a suitable instrument holder.
- Fractional pre-vacuum (4 times).
- Sterilisation temperature 134°C.
- Hold time 5 minute (full cycle).
- Drying time 10 minutes.

Steam sterilisation in the fractional vacuum process

(Machine complies with EN 13060, Class B)

- Instruments wrapped in suitable packaging for sterilising goods (see DIN 58952/53 and EN 868).
- Fractional pre-vacuum (4 times).
- Sterilisation temperature 134°C.
- Hold time 5 minute (full cycle).
- Drying time 10 minutes.

6 Storage

- Store protected from dust, moisture and recontamination and observe the storage periods (see DIN 58953).

! Note !

New instruments supplied non-sterile must be processed before initial use.

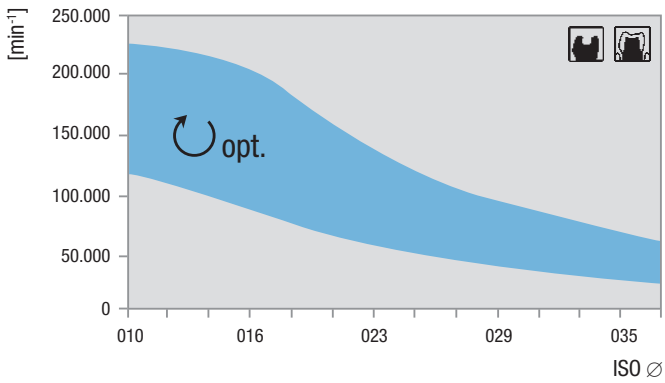
Steel instruments are unsuitable for both mechanical preparation and the steam steriliser. They should be replaced by corresponding tungsten-carbide instruments.

Single-use products (identified on the packaging with ) are not approved for reuse.

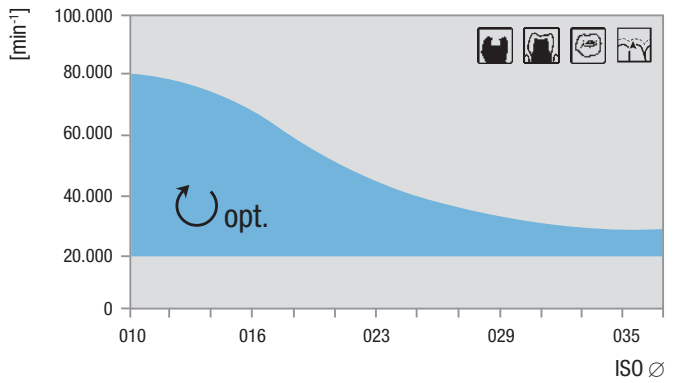
FlexiSnap mandrels are reusable, can be used in a CDM and are steam sterilisable. **FlexiSnap discs** are single-use products and can **only be manually** (unsuitable for CDM and steam steriliser) **disinfected** using a suitable disinfectant (e.g. HELVEMED Instrument Forte)

Colour-anodised aluminium units (e.g. Bur block 40500 to 40580 and Retopin mandrel) lose their colour during use of standard cleaning procedures and in a CDM. Cleaning and disinfectant agents, which are specifically designed for these materials (e.g. HELVEMED Instrument Thermo EC), should be used during preparation.

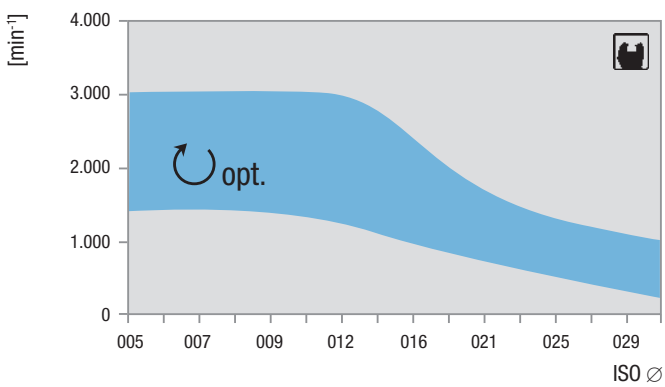
Präparieren (Diamantinstrumente)
Preparation (Diamond instruments)
Préparer (instruments diamantés)

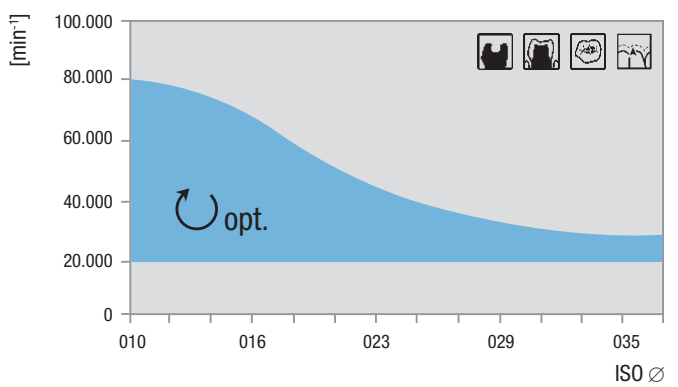
Finieren (Diamantinstrumente)
Finishing (Diamond instruments)
Finir (instruments diamantés)

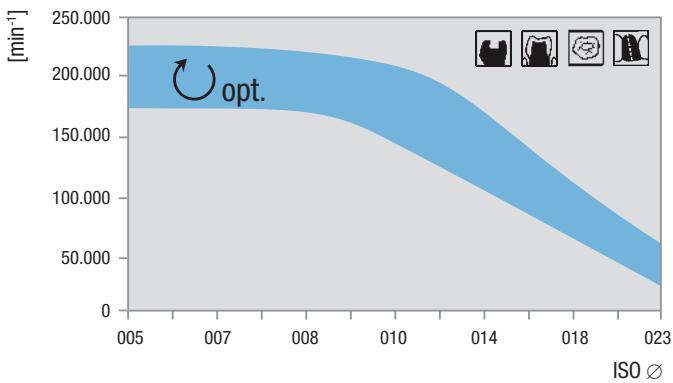
Exkavieren (Hartmetall- u. Stahlinstrumente)
Excavating (Tungsten carbide and Steel instruments)
Excaver (instruments en carbure et acier)

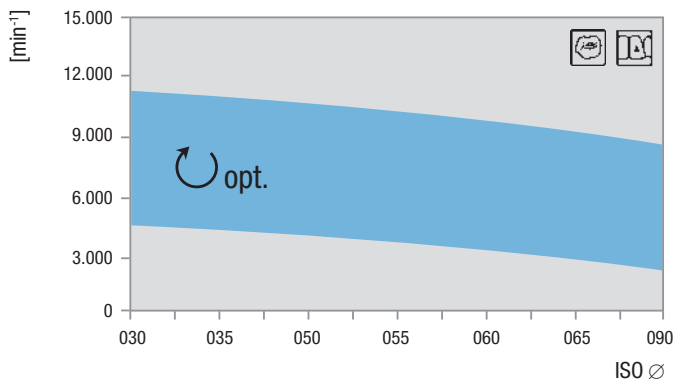
Finieren (Hartmetallinstrumente)
Finishing (TC instruments)
Finir (instruments en carbure)

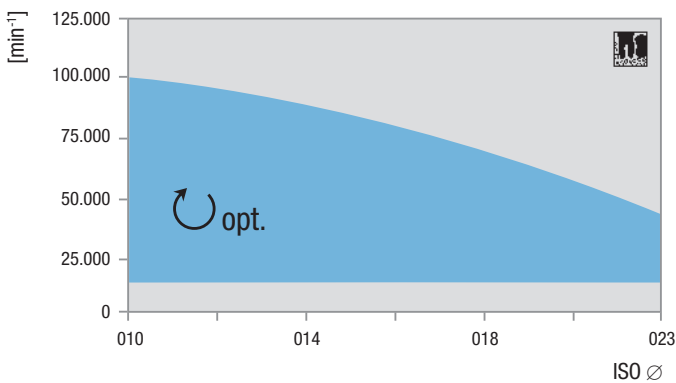
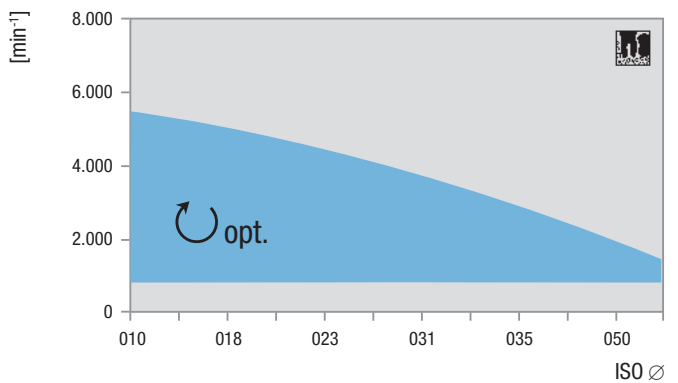
Präparieren & Trennen (Hartmetallinstrumente)
Preparation & separating (TC instruments)
Préparer & séparer (instruments en carbure)

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