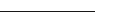
REF NTI Logic Set 5

836KR-010M -FG	
836KR-010F -FG	
836KR-012M -FG	
836KR-012F -FG	
837KR-012M -FG	
837KR-012F -FG	
837KR-014M -FG	
837KR-014F -FG	
850-012M -FG	
369-025M -FG	

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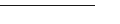
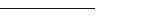
Preparation of teeth for full and metal-ceramic crowns

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
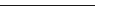
Preparation of teeth for full cast or composite veneer crowns

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Axial Reduction Instruments by Dr Mark J. Caldwell

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MADC Depth Markers by Dr Mark J. Caldwell, USA

Depth marking for all areas of crown and bridgework.

All-porcelain and CAD/CAM produced restorations place high demands on dental preparations. The success of a restoration depends to a high extent upon the compliance of the minimal wall thickness of the restoration (Teamwork 6/2007; Prof. Dr. Frankenberger et.al.).

With the „Caldwell“ (acc. to Dr Mark J. Caldwell) diamond instruments, these requirements can be safely and easily achieved.

With these seven instruments it is possible to define the prepared margin at the depths of 0,3 mm - 0,6 mm; 1.0 mm; 1.5 mm and 2.0 mm. The MADC instruments have ring markings stating each individual depth. This way you can see the differences even when in the bur stand. With these instruments all preparations are possible, with any technically or physically required depth.

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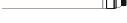
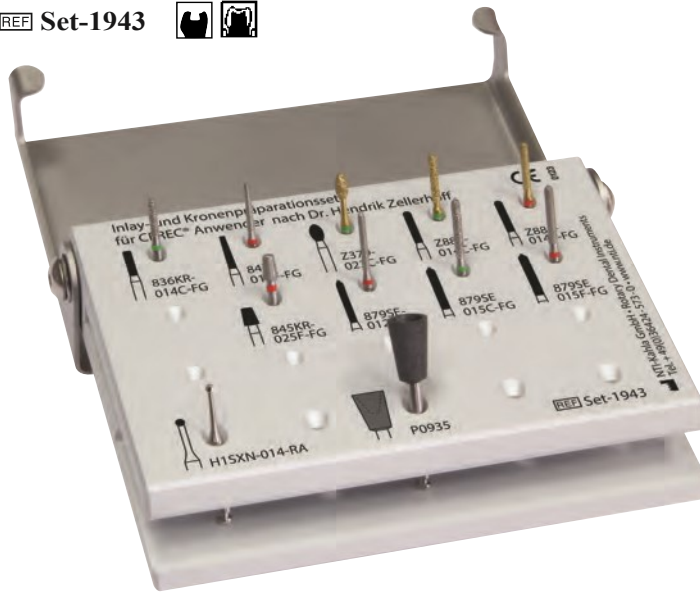
Inlay and crown preparation set for CEREC® by Dr Hendrik Zellerhoff

Dr Zellerhoff has been working in his own practice since 2006 and has specialised in digital dentistry. In addition to working in his practice, he acts as an international consultant for CEREC®. He is also a speaker for the DDA (Digital Dental Academy) Berlin and the DGCZ (German Society for Computer-guided Dentistry). The combination of instruments in Set-1943 corresponds to the NTI concept of „facilitating work with innovative instruments“ and the CEREC® system concept of „Fewer treatment steps, fewer sources of error“.

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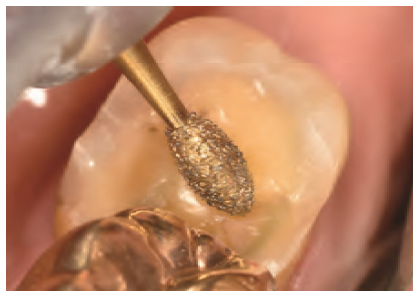
Smoothing of edges to avoid tensile stress on the ceramic.

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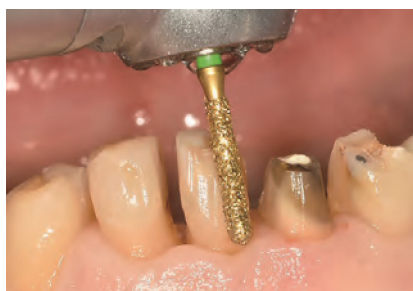
Substance reduction of the chamfer in well visible situations

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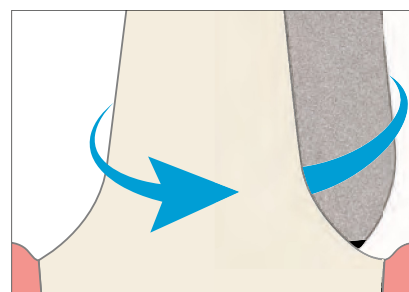
Congruently shaped finisher for the crown stump

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Preparation Set for CEREC® Veneers designed by Dr. Klaus Wiedhahn

Set for preparing CEREC® veneers and final finishing developed in collaboration with Dr. Klaus Wiedhahn.

The coordinated instruments for preparing veneers guarantee optimum preparation and consequently accurately fitting restorations.

The coarse grit size of the preparation diamonds produces rapid, time-saving reduction of the tooth structure.

The fine instruments create optimum preparation margins, which are required for accurate recording with an intraoral camera and facilitating digital data processing. This guarantees a gap-free fit of the veneer at the natural tooth.

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An instrument set for perfect CEREC® crown and bridge preparation designed by Dr Klaus Wiedhahn.

The set contains all the essential instruments for crown and bridge preparation, from removing old crowns to safe exposure of the cervical preparation margin using the tissue trimmer.

The set includes coarse and fine preparation instruments. These facilitate rapid preparation for crowns and bridges fabricated using CAD/CAM.

The low number of instruments simplifies the preparation sequence.

The CeraGlaze polishers in the set create the same surface texture as that produced following standard glaze firing, without any risk to the marginal seal of the restoration or delayed crack formation.

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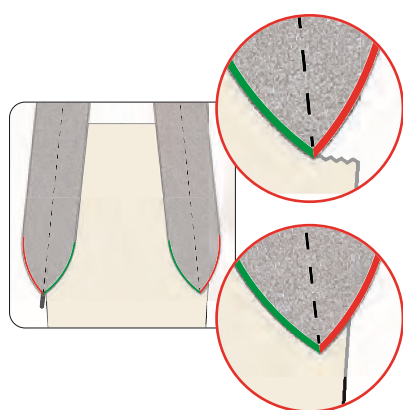
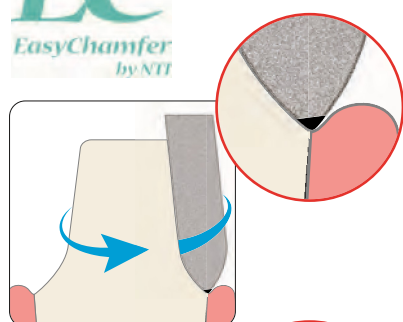
With fully diamonded preparation instruments, there is always the risk of creating so-called negative grooves or unevenness in the preparation as of the mid-section of the instruments.

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Instruments with a guide pin only permit a certain circular preparation depth.

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
The EasyChamfer with its partially diamonded instrument tips automatically precludes these risks. The instrument tip is actively involved in preparation.

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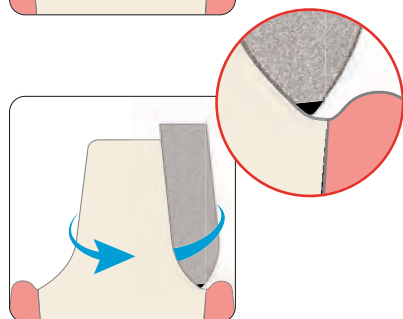
Within the preparation, the instrument supports itself cervically as the non-diamonded tip can run on the chamfer without damaging the chamfer. There is no abrasion at the tip of the instrument.

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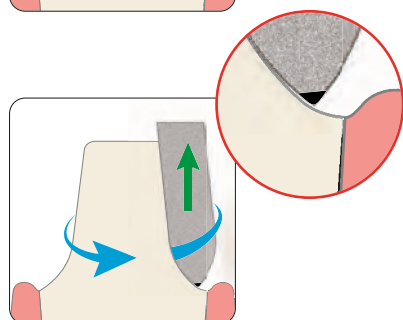
By intentionally raising the instrument during the preparation movement (blue arrow) in occlusal direction (green arrow), abrasion is activated and deeper shaping of the chamfer is always possible. Subsequently, the instrument tip is actively involved in the preparation again.

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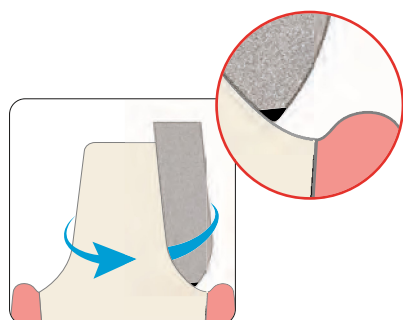By applying this technique, the preparation can be made as wide as desired with just a single instrument without damaging the gingiva. The position of the preparation margin can be changed further to gingival or subgingival at any time.

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ALL IN ONE All-Porcelain Set

The compact, high quality set for all-porcelain restoration preparation.

The most recent preparation requirements for modern adhesive dentistry, including CAD / CAM technology, have been accounted for.

The instruments facilitate effective and time-saving preparation with minimal change of instruments: from inlays, onlays and partial crowns to crowns and bridges with a modified chamfer (CAD / CAM).

The ALL IN ONE set is also shaped for all-porcelain veneers.

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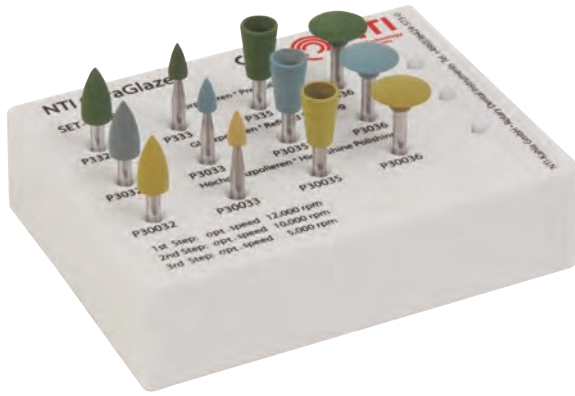
Z-Cut diamond instruments

Ideal for the preparation of zirconia abutments and separation of zirconia crowns.

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NTI CeraGlaze RA

The NTI CeraGlaze is produced from pure natural rubber. The diamond grit is optimally distributed throughout the binder thanks to the special production procedure. The binder absorbs the heat generated during polishing which prevents damage to the delicate ceramic restorations. The combination of the binder and the selected natural diamond grit generates an unmatched gloss with a depth of roughness that corresponds to a final firing.

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The C·E·O Set-1934 was developed for the preparation of occlusal composite fillings.

Compared to classical preparation, this makes faster, safer, more ergonomic and more stress-free work possible. The cavity can be prepared within 30 seconds using the special NTI InPrep diamond instrument. The passive tip reliably prevents penetration of the pulp chamber.

The filling surface is processed with the NTI LazyBur diamond instrument. In contrast to the classic shapes used, such as the flame, the LazyBur enables faster working without lateral oral or buccal deflection. This avoids ergonomically unfavourable positions for the dentist and the dental assistant, especially with prep-work on the upper molars. The treatment is also less stressful for the patient as the cheek does not have to be stretched buccally. After finishing, polishing is performed with the NTI Unique diamond impregnated polishers. The diamond impregnation guarantees excellent polishing results even on very hard nanofilled composites. The exact use of the instruments is shown in the video.

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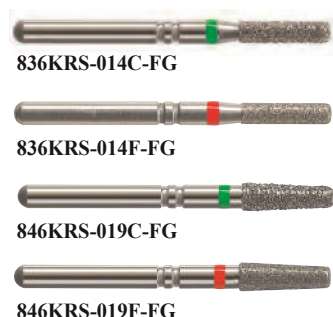
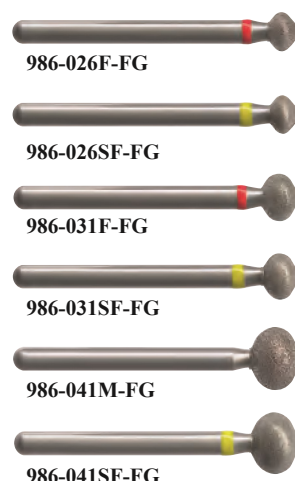
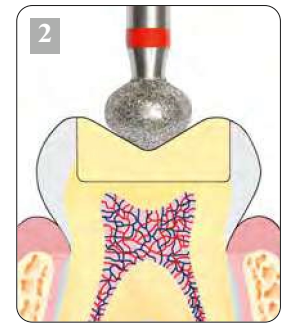
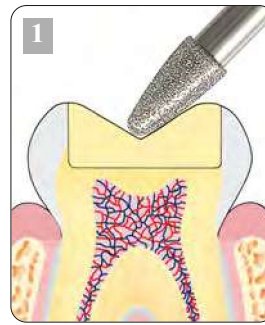
Occlusal shaping of composite restorations in the posterior region with the "LAZYBUR" designed by Dr. Michael Beier

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1. Treatment to-date

In the case of conventional instruments with a "pointed shape", a strong lateral tilt is usually necessary, which in turn leads to unphysiological stretching of the cheek.

2. Treatment with the LazyBur

The entire movement sequence is guided vertically without tilting the instrument, owing to the special geometry of the LazyBur.

Your advantages at a glance

- smooth, clear treatment process
- minimal movement manoeuvres
- no need for changing your grip or overstretching of the wrist or spreading of the cheek
- ergonomic handling
- simplified holding, suction and finishing
- faster and more efficient working
- relaxed patient and satisfied customer

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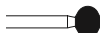
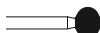
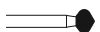
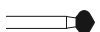
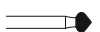
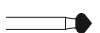
Procedure for finishing

Conceptual orientation always follows the central fissure.

The LazyBur is only guided vertically.

Tilting the drive as with other instruments is no longer necessary to design the different cusps.

One instrument for the complete design of occlusal surfaces.

Perfect and fast contours on all filling materials.

Three sizes allow shaping of all premolars and molars.

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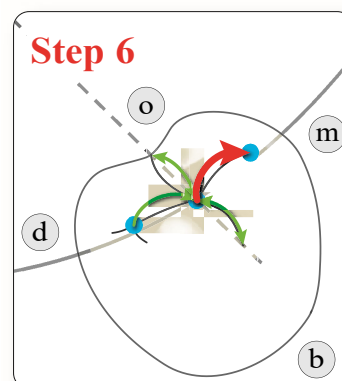
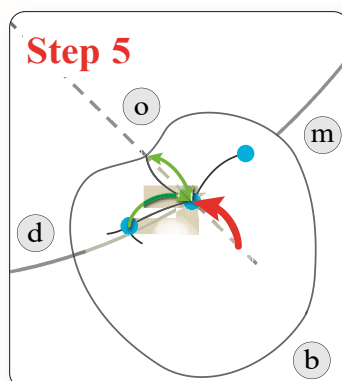
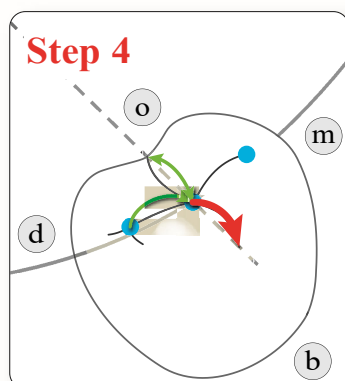
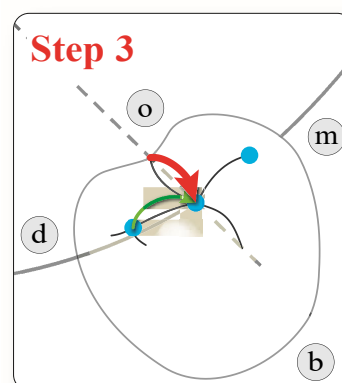
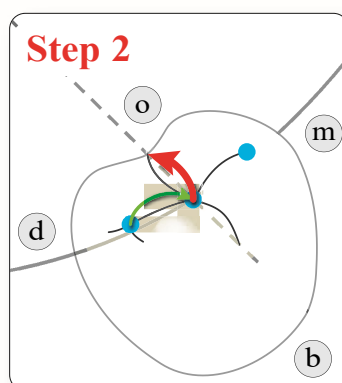
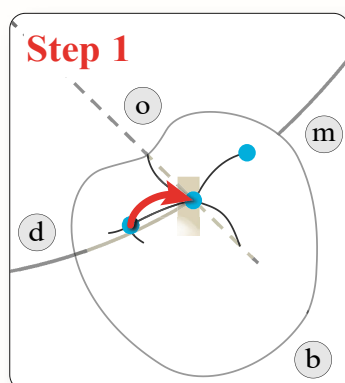
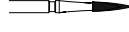
Universal Composite Set

includes all necessary rotary instruments for Caries elimination, Removal of old fillings, Cavity preparation classes I, II, III, IV and V, Contouring, Finishing and Polishing of Composite fillings.

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Removing decay dentine. Opening and widening the cavity.

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Removing old amalgam or composite fillings with carbide burs.

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
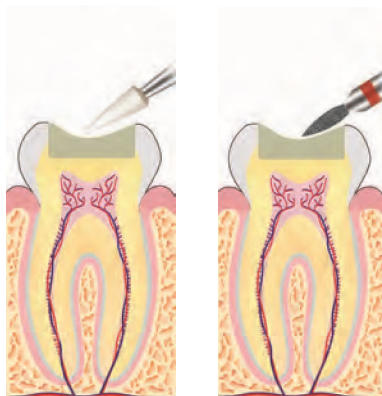
After hardening the surface of the composite filling is trimmed and contoured.

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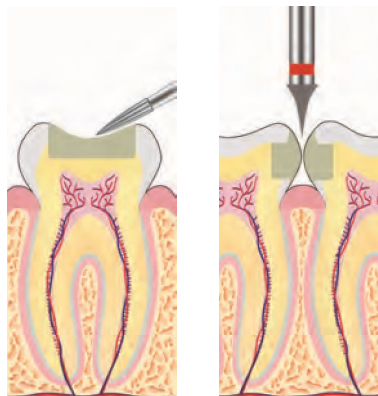Finishing the surface with finishing carbide burs.

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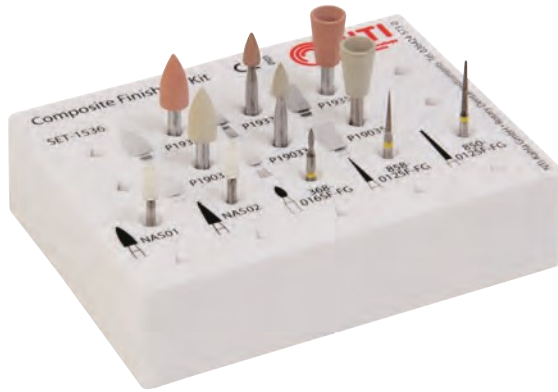
Polishing with NTI Unique, diamond impregnated one stage polishers, for best results use with water ! Abrasive polishing at a speed of 15,000 rpm; glaze polishing at 10,000 rpm ; work with slight pressure.

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Composite Finishing Kit

This assortment is for finishing and polishing all microfilled composites. Produces a high lustre quickly.

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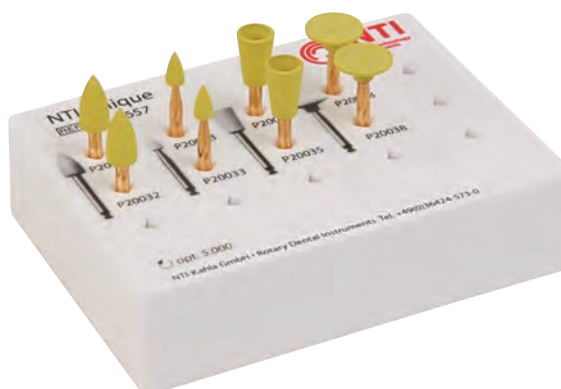
NTI DiaGloss Set

DiaGloss is a diamond-impregnated polisher with a rubber binder. The binder absorbs heat which prevents damage to the composites. Using the red polisher enables teeth to be polished without being damaged by overheating.

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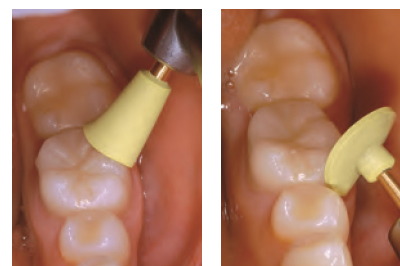
NTI Unique Set


Natural diamonds in the polisher ensure exceptional polishing results on all composites. With Unique polishers, preliminary polishing can be carried out at 10,000 rpm and the high gloss polishing can be carried out at 5,000 rpm. The natural rubber binder prevents overheating of the filling materials thanks to its ability to absorb heat.

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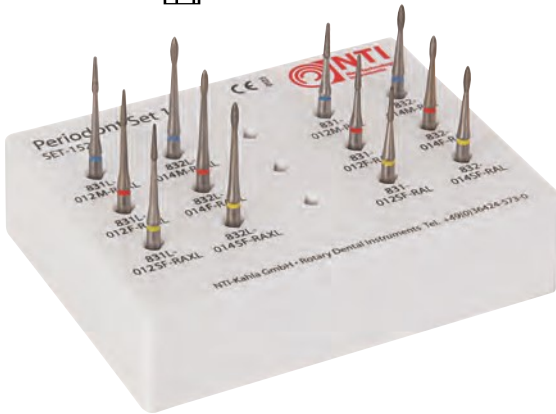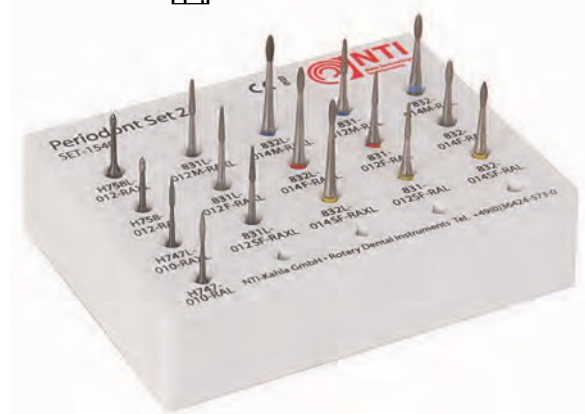
Periodont Set 1

(Periodontal Diamond Set) Two different shapes with two different shank lengths for trimming root surfaces.

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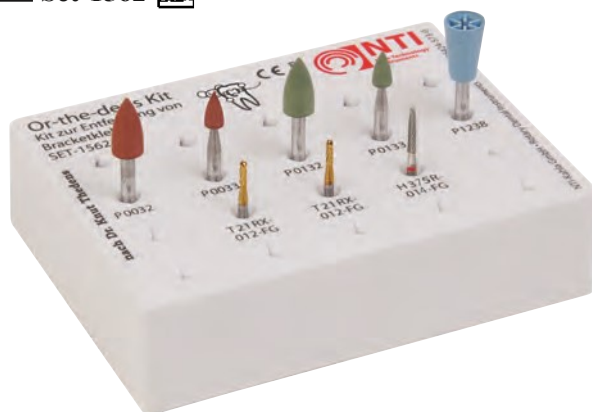
Periodont Set 2

(Periodontal Diamond and Tungsten Carbide Set)
Two different shapes with two different shank lengths, one each tungsten carbide and diamond, for trimming root surfaces.

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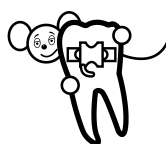
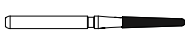
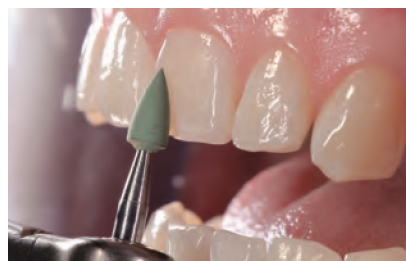
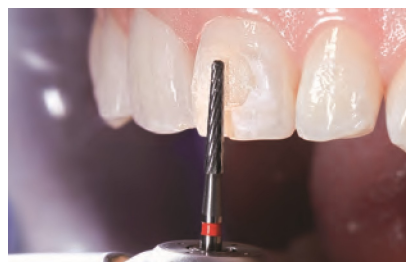
Or-the-dens Kit by Dr Knut Thedens.

The system was developed for removing bracket bonding resin. The tungsten carbide bur is coated in titanium nitrite and ensures that the resin is removed as quickly as possible. The coating has a particularly smooth surface which prevents the teeth clogging up. The last traces of resin are removed using the finishing bur. Damage to the enamel surface is precluded with these instruments. The polishing produces perfect surfaces without retention sites for bacteria.

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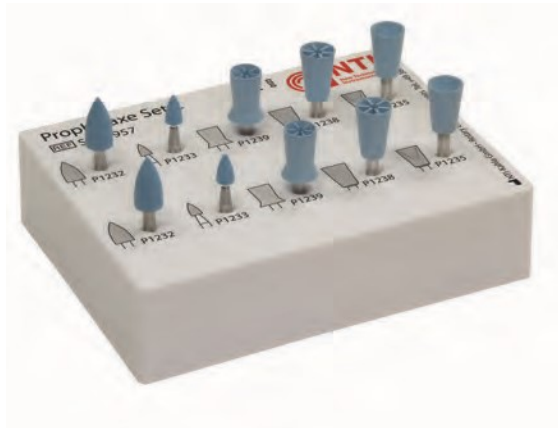
Prophylaxis Set 1, Set-1956

A complete set for cleaning and polishing teeth, amalgam and composite fillings and precious metal alloys.

Abrasive material is integrated in the blue prophylactic polishers. Polishing paste is not therefore required. With very stubborn deposits, polishing paste can be used to increase the polishing capacity.

The NTI EpsiPol is suitable for use on all precious and semi-precious metal alloys as well as amalgams. The brown prepolisher removes scratches and the green polisher produces a perfect lustre.

The yellow Unique polisher is a diamond-coated one-step polisher for composite fillings.

The selected diamond grit allows all types of composite to be polished with a single polisher.

Prophylaxis Set 2, Set-1957

This prophylactic set is a complete set just for cleaning teeth.

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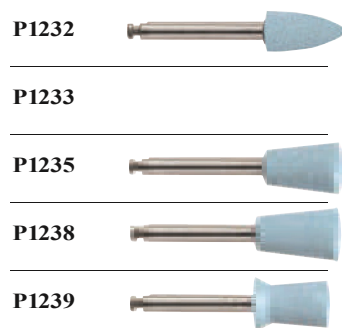
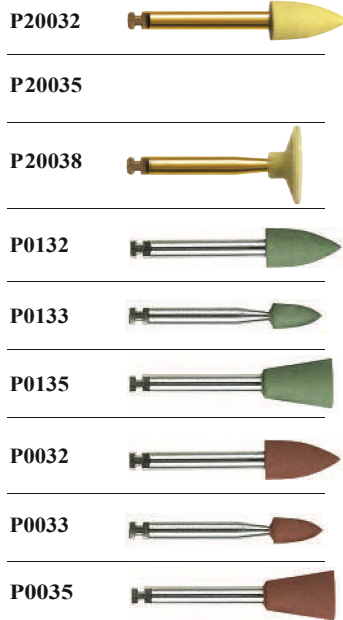
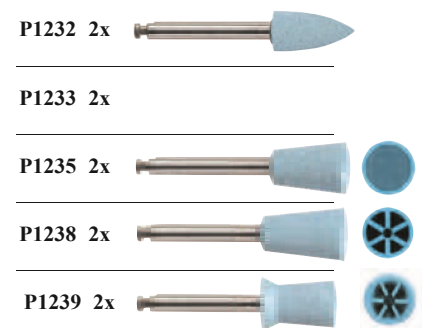
Instrument set for universal preparation for implant placement.

The set developed by Dr. Damien Carrotte / Villeurbanne France, is used for reliable determination of the implant axis.

Initially, the bone is deepened and cleaned up using an Allport instrument H141AX, diameter ISO 050. The blade geometry and coating of the instrument provide high cutting efficiency and long service life. The first drilling cycle is then completed using the triangular centering drill RF186LM. This very thin, sharp instrument does not drift and permits reliable determination of the axis, even with difficult situations. The markings facilitate accurate orientation regarding the depth of the drill hole.

Following this initial determination of the axis, the drill hole is extended by the pilot drill RF210LM1 with the larger diameters ISO 020, ISO 028 or ISO 035.

The markings facilitate accurate orientation regarding the depth of the drill hole. The sharpness of the blades enables to collect bone chips for autografts.

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